

ENMET Corporation
PO Box 979
Ann Arbor, MI 48106-0979

MEDAIR 2200
Operation and Maintenance
Manual

Manual part number
80002-042
MCN-439, 08/20/10

Table of Contents

1.0 INTRODUCTION	1
1.1 Unpack	1
1.2 Check Order	1
1.3 Serial Numbers	1
2.0 INSTRUMENT FEATURES	2
2.1 Exterior Features	2
2.2 Display Panel Features	2
2.3 Circuit Board Features	4
3.0 INSTALLATION	5
3.1 Mounting of Instrument	5
3.2 Sample Air Supply	6
3.3 Power Supply	6
3.4 Outputs	6
3.4.1 Relay Contacts	7
3.4.2 Optional 4-20mA Outputs	8
3.5 Installation Verification	8
4.0 OPERATION	9
4.1 Normal Operation Condition.....	9
4.2 Alarm Set Points	9
4.3 Alarm Latching or Differential Settings	10
4.4 Audio Defeat	10
4.5 Display	10
4.6 Operational Menu	11
4.7 Fault Indications	12
4.7.1 Low Flow Indication	12
4.7.2 Other Fault Indications.....	12
4.8 Dew Point Sensor Response.....	12
4.9 Hydrocarbon Sensor Response	12
5.0 MAINTENANCE	13
5.1 Cleaning Instructions	13
5.2 Maintenance Menu.....	13
5.2.1 Accessing Maintenance Menu.....	13
5.2.2 Maintenance Menu Flow Chart	14
5.3 Calibration for CO, O ₂ , HC and CO ₂ (Gas Channels).....	15
5.3.1A Low Cal/ZeroCal Adjust	16
5.3.1B High Cal/SpanGas Adjust	16
5.3.2 Set 4 –20mA Transmitter Scale	17
5.3.3 Set Alarm Points.....	18
5.3.4 Set Alarm Delay	20
5.3.5 Relay Configuration.....	21
5.3.6 Failsafe Configuration	22
5.3.7 Set Output Span Range.....	22
5.3.8 Set New Password.....	23
5.3.9 Exit Maintenance Menu	23
5.4 Sensor Replacement.....	24
5.4.1 Oxygen / Gas Sensor.....	24
5.4.2 Calibration/Sensor Replacement.....	24
5.4.3A Low Cal/ZeroCal Adjust	26
5.4.3B High Cal/SpanGas Adjust	26
5.5 Dew Point Sensor.....	27
5.6 Flow Control Orifice / Pre-filter	27
6.0 TECHNICAL DATA AND SPECIFICATIONS	28
7.0 REPLACEMENT PART NUMBERS	29
7.1 ENMET part numbers for sensors and replacement parts.....	29
7.2 ENMET part numbers for Calibration equipment:	29
8.0 WARRANTY	30
APPENDIX A: CO CHARACTERISTICS	31
APPENDIX B: GAS IONIZATION POTENTIALS	32

List of Figures

FIGURE 1: EXTERNAL FEATURES OF THE MEDAIR 2200.....	3
FIGURE 1A: SAMPLE AIR HOSE AND REGULATOR	3
FIGURE 2: MEDAIR 2200 INTERIOR FEATURES	4
FIGURE 3: MEDAIR 2200 MOUNTING DIMENSIONS.....	5
FIGURE 2A: RELAY, INPUT AND OUTPUT TERMINALS	6
FIGURE 4: MEDAIR 2200 OPERATIONAL DISPLAY	9
FIGURE 5: MEDAIR 2200 OPERATION MENU FLOW CHART	11
FIGURE 6: MEDAIR 2200 MAINTENANCE MENU FLOW CHART.....	14
FIGURE 7: CONNECTION OF CALIBRATION GAS CYLINDER	16
FIGURE 9: LOCATION OF GAS SENSOR AND DEW POINT MANIFOLDS.....	24
FIGURE 7: CONNECTION OF CALIBRATION GAS CYLINDER	26
FIGURE 10: CARBON MONOXIDE CONCENTRATION	31

List of Tables


TABLE 1 : RELAY FAILSAFE SETTINGS	7
TABLE 2: SENSOR OUTPUT.....	8
TABLE 3: TYPICAL FACTORY ALARM SET POINTS.....	9
TABLE 4: FACTORY SET GAS ALARMS DELAY	20

Reference Information:

NOTE: [important information about use of instrument]

CAUTION: [affects equipment – if not followed may cause damage to instrument, sensor etc...]

WARNING: [affects personnel safety – if not followed may cause bodily injury or death.]

 Attention / Warning

 Earth Ground

1.0 Introduction

The **MEDAIR 2200** is a compressed air monitoring instrument that measures and detects certain hazards in medical air supply systems. The instrument is available with up to 4 internal sensors. Available sensors include, but are not limited to carbon monoxide (CO), carbon dioxide (CO₂), Hydrocarbons (HC) for variations in the oxygen (O₂) content and for dew point. The sensors can be used alone or up to four sensors can be used together. In the instrument, a sample of the compressed air is passed over each sensor and the resulting electrical outputs are used to evaluate the air for the target gases.

Some features of the instruments are as follows:

- continuous monitoring of the sample air
- continuous LCD display of gas and vapor concentrations
- menu driven operational and maintenance controls
- menu driven calibration procedure
- audio and visual alarms indicate unsafe conditions
- alarm relay contacts available on terminals
- a fault relay and visual fault alarm
- low air flow fault indication and display
- alarm acknowledgement capability including audio defeat
- mA outputs for each target gas

Hydrocarbons (HC) are limited to gases with an ionization potential of 10.6 eV or less. See Appendix B for a list of gases and IPs.

NOTE: *All specifications stated in this manual may change without notice.*

1.1 Unpack

Unpack the **MEDAIR 2200** and examine it for shipping damage. If such damage is observed, notify both **ENMET** customer service personnel and the commercial carrier involved immediately.

Regarding Damaged Shipments

NOTE: **It is your responsibility to follow these instructions. If they are not followed, the carrier will not honor any claims for damage.**

- This shipment was carefully inspected, verified and properly packaged at our company and delivered to the carrier in good condition.
- When it was picked up by the carrier at **ENMET**, it legally became your company's property.
- If your shipment arrives damaged:
 - Keep the items, packing material, and carton "As Is." Within 5 days of receipt, notify the carrier's local office and request immediate inspection of the carton and the contents.
 - After the inspection and after you have received written acknowledgment of the damage from the carrier, contact **ENMET** Customer Service for return authorization and further instructions. Have your Purchase Order and Sales Order numbers available.
- ENMET** either repairs or replaces damaged equipment and invoices the carrier to the extent of the liability coverage, usually \$100.00. Repair or replacement charges above that value are your company's responsibility.
- The shipping company may offer optional insurance coverage. **ENMET** only insures shipments with the shipping company when asked to do so in writing by our customer. If you need your shipments insured, please forward a written request to **ENMET** Customer Service.

Regarding Shortages

If there are any shortages or questions regarding this shipment, please notify **ENMET** Customer Service within 5 days of receipt at the following address:

ENMET Corporation
680 Fairfield Court
Ann Arbor, MI 48108
734-761-1270 734-761-3220 Fax

1.2 Check Order

Check the contents of the shipment against the purchase order. Verify that the **MEDAIR 2200** is received as ordered. If there are accessories on the order, ascertain that they are present. Check the contents of calibration kits. Notify **ENMET** customer service personnel of any discrepancy immediately.

1.3 Serial Numbers

Each **MEDAIR 2200** is serialized. These numbers are on tags on the equipment and are on record in an **ENMET** database.

2.0 Instrument Features

2.1 Exterior Features




The exterior of the instrument is shown in **Figure 1**. The exterior features are as follows:

Feature	Description
Enclosure	An engineered thermoplastic box, approximately 10x8x6, with a clear hinged front cover.
Input Port	The entrance for the air sample and calibration gas. The quick release fitting mates with one on the calibration adapter.
Front Cover Latch	A quick-release latch that holds the clear front cover in place, and is capable of being padlocked if desired.
Audio Alarm	A loud horn activated by certain alarm conditions.
Mounting Flanges	Flanges with holes for mounting the enclosure to a vertical surface.
Sample Air Hose	A five foot long hose to conduct a sample of the air from the source to the instrument. The hose has a Female quick release fitting and Male ¼" NPT fitting. See Figure 1A .

NOTE: When connecting to a standard 55 PSI USA Medical air system, Regulator is Not required.

2.2 Display Panel Features

The display panel, shown in **Figure 1**, is viewed through the clear front cover of the enclosure, and is accessed by opening the cover. Features are as follows:

Feature	Description
Display	A 2 line, 16 character per line, LCD with backlight. The numerical values of gas concentrations, and other information are displayed.
Visual Alarms & Indicators	On either sides of the display: A red alarm LED for each sensor installed in the instrument, Low level alarm. The top center of the panel: A red alarm LED for all sensors installed in the instrument, High level alarm. Near the center of the panel: A green power indicator LED A red fault alarm indicator LED
Pushbutton Switches	There are three of these, located near the center of the panel; they are yellow rectangular membrane switches. They are:
• OPTION Switch	The top left switch. 
• SELECT Switch	Directly to the right of the OPTION switch. 
• Audio Defeat / Alarm Acknowledge Switch	Directly below the OPTION switch. 

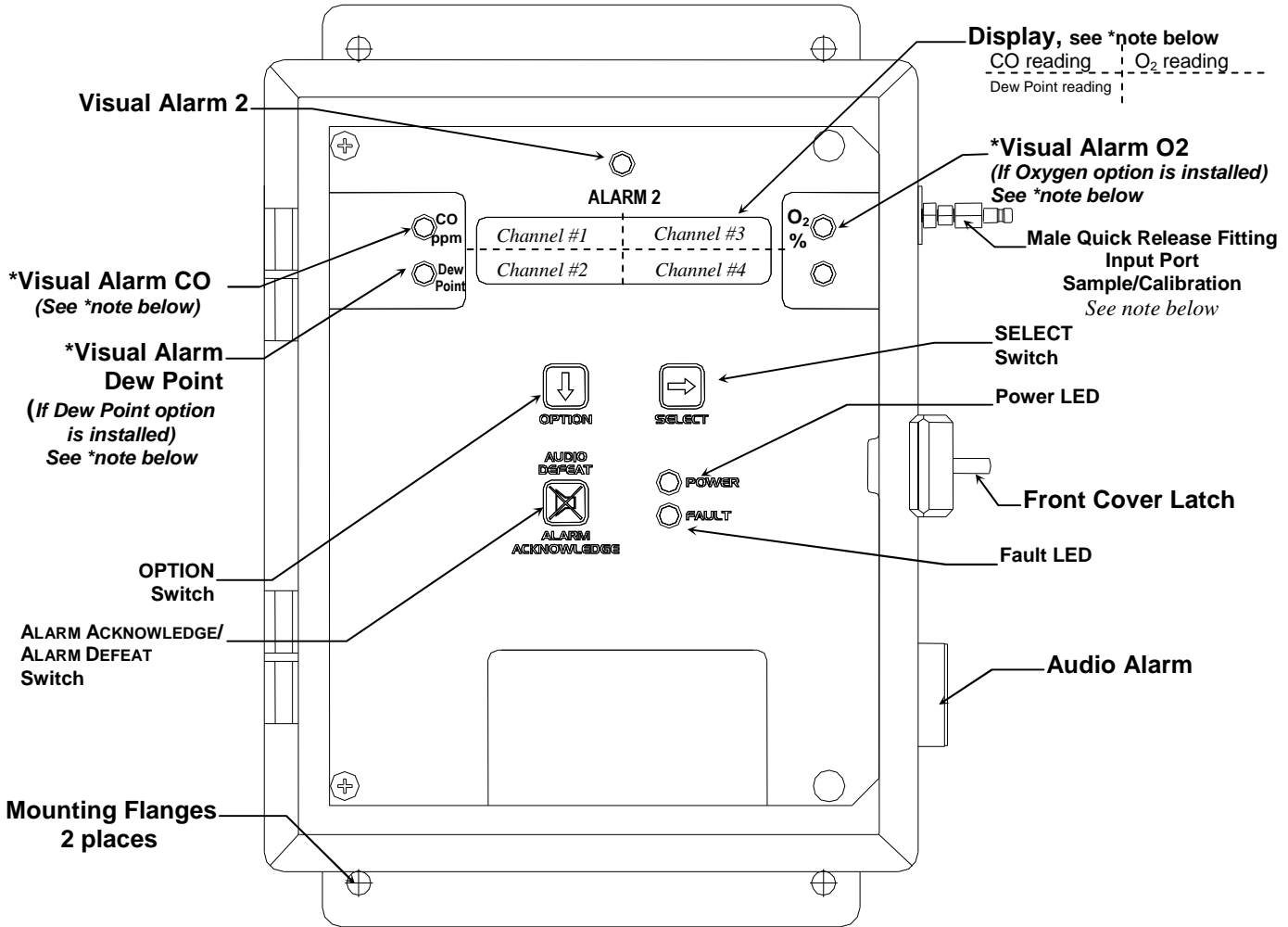


Figure 1: External Features of the MEDIAIR 2200

*** NOTE: Typical gas reading & alarm locations, depending on instrument configuration, alarms & readings may be in alternate locations**

NOTE: When connecting to a standard 55 PSI USA Medical air system, Regulator is Not required.

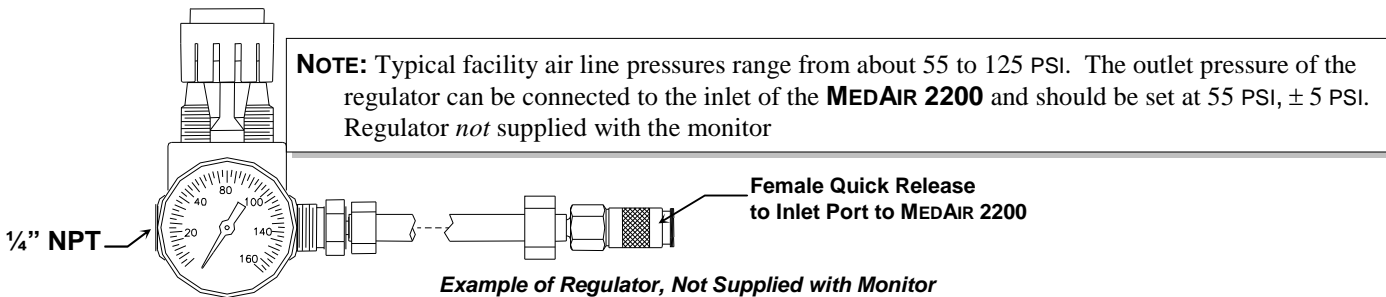
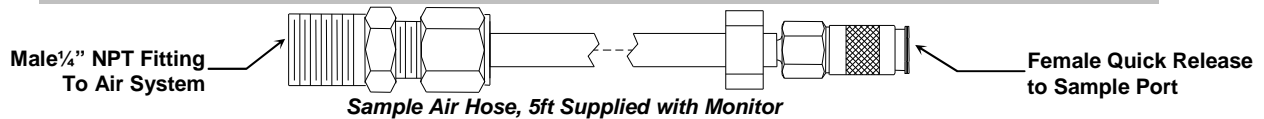


Figure 1A: Sample Air Hose and Regulator

2.3 Circuit Board Features

The Display Panel is hinged on the left and is released by unscrewing the 2 screws located in the right corners. After releasing the panel, it is swung to the left, exposing the interior of the enclosure. The Circuit Board is mounted at the back surface of the enclosure interior. Features are shown in **Figure 2**.

Feature	Description
Relay Terminals	This group of terminals is located at the left side of the Circuit Board. For the contacts for each of four alarm relays, and for the contacts of a fault relay.
Output Terminals	One 4-20mA output per active channel. 2 channels/outputs per connector.
Sensor Manifold	The sample manifold, the carbon monoxide, carbon dioxide and oxygen sensors are located under this housing.
Dew Point Manifold	The dew point sensor is installed into this housing.

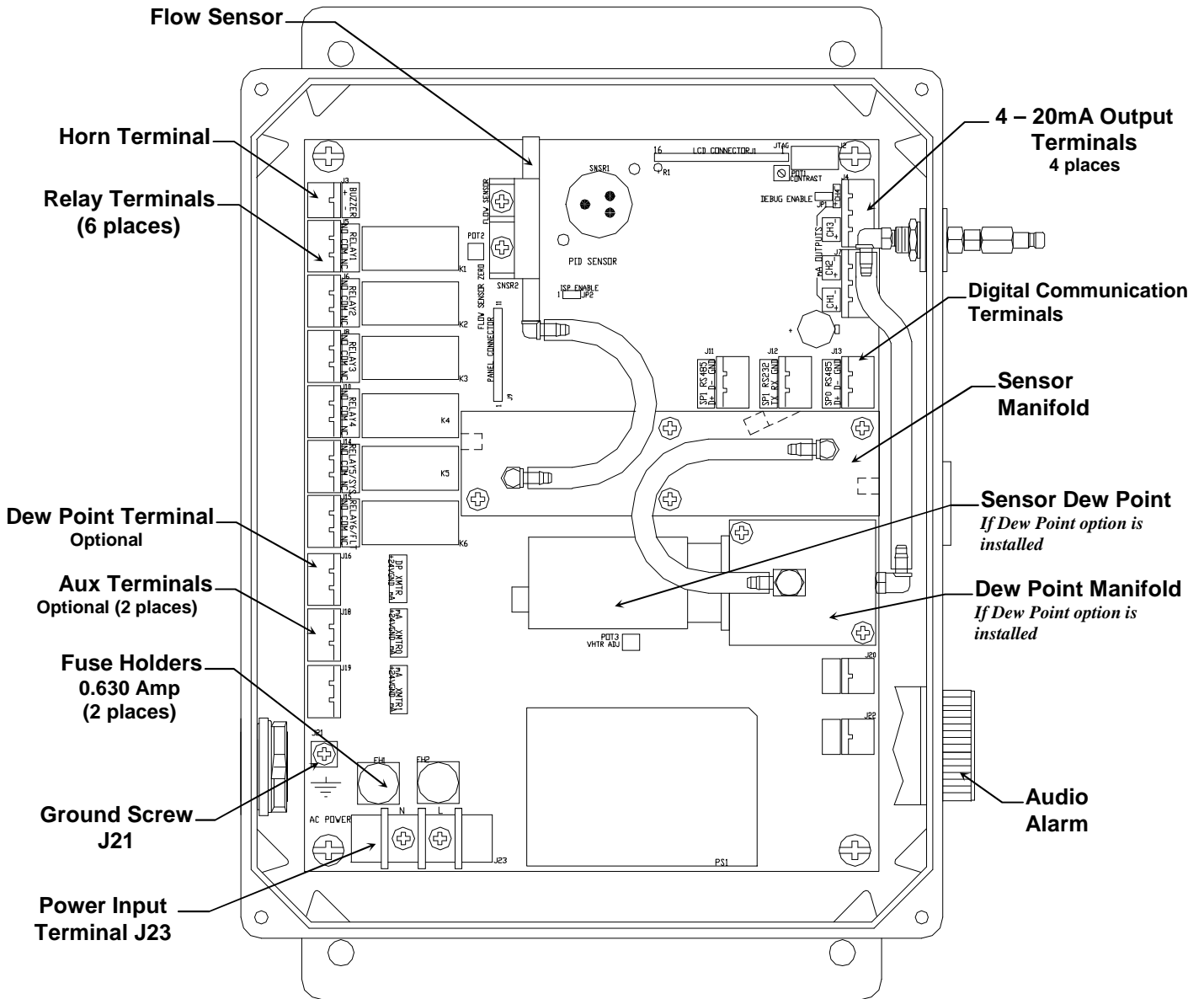


Figure 2: MEDIAIR 2200 Interior Features

3.0 Installation


3.1 Mounting of Instrument

The **MEDIAIR 2200** should be located near the pipe or tank containing the air to be monitored, and upstream from where the air is being used. The **MEDIAIR 2200** must be installed such that it samples the compressed air before it reaches the users.

Mount the instrument on an appropriate vertical surface using the mounting flanges provided. Avoid areas with excessive vibration or temperature extremes. The holes in the flanges are 0.31 inch in diameter and form a 6 x 10.75 inch rectangle. See **Figure 3**.

It is recommended to use #8 drywall anchors and screws for mounting the **MEDIAIR 2200** to a drywall/sheetrock surface.

**Mounting Holes
0.31" dia. 4 places**

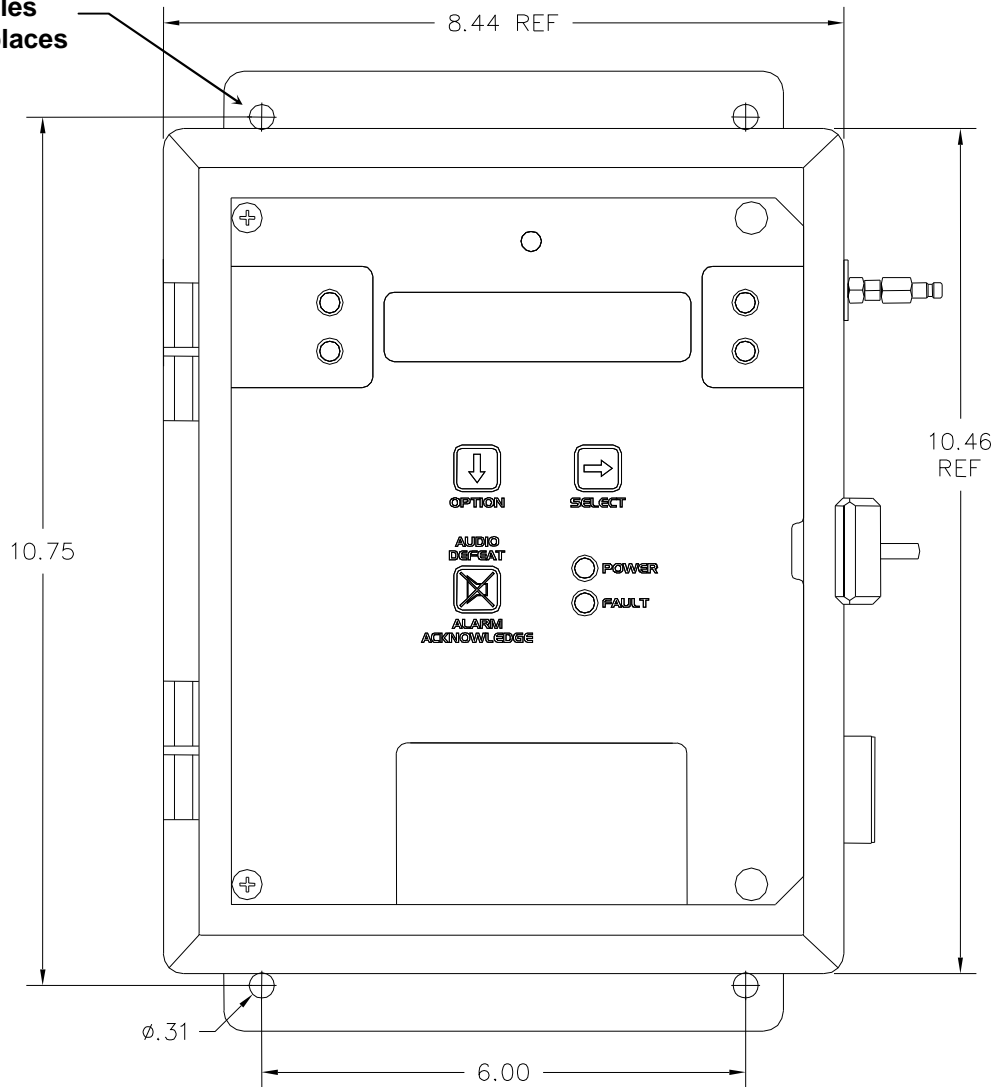

Dimensions are in inches.

Figure 3: MEDIAIR 2200 Mounting Dimensions

3.2 Sample Air Supply

Tap the pipe or tank containing the breathing air and use appropriate fittings to connect the sample input hose. The instrument is designed to operate from an air supply pressure 55 PSIG.

The sample air exits the instrument from the hole plug located on the bottom of the enclosure. Take care not to obstruct this exit port.

3.3 Power Supply

The input power can vary from 100 to 240VAC, 50/60 Hz. Mains power should be connected to the Power Input Terminal **J23** and the ground screw **J21**. See **Figure 2** for location.

! WARNING: Continuous gas detection and alarm systems become inoperative upon loss of primary power.

Upon supplying air and power to the instrument:

- The green power on LED is lit.
- The display backlight is lit, and instrument will step through a start-up sequence: unit serial number, software revision and gases monitored may be shown on the display.

The instrument may go into alarm briefly, but the sensors stabilize quickly. If the instrument persists in alarm, acknowledge the alarm by pressing the **AUDIO DEFEAT / ALARM ACKNOWLEDGE** switch. If alarm persists longer than 30 minutes, call **ENMET** customer service personnel.

3.4 Outputs

Two types of alarm outputs are available, relay contacts and 4-20mA outputs.

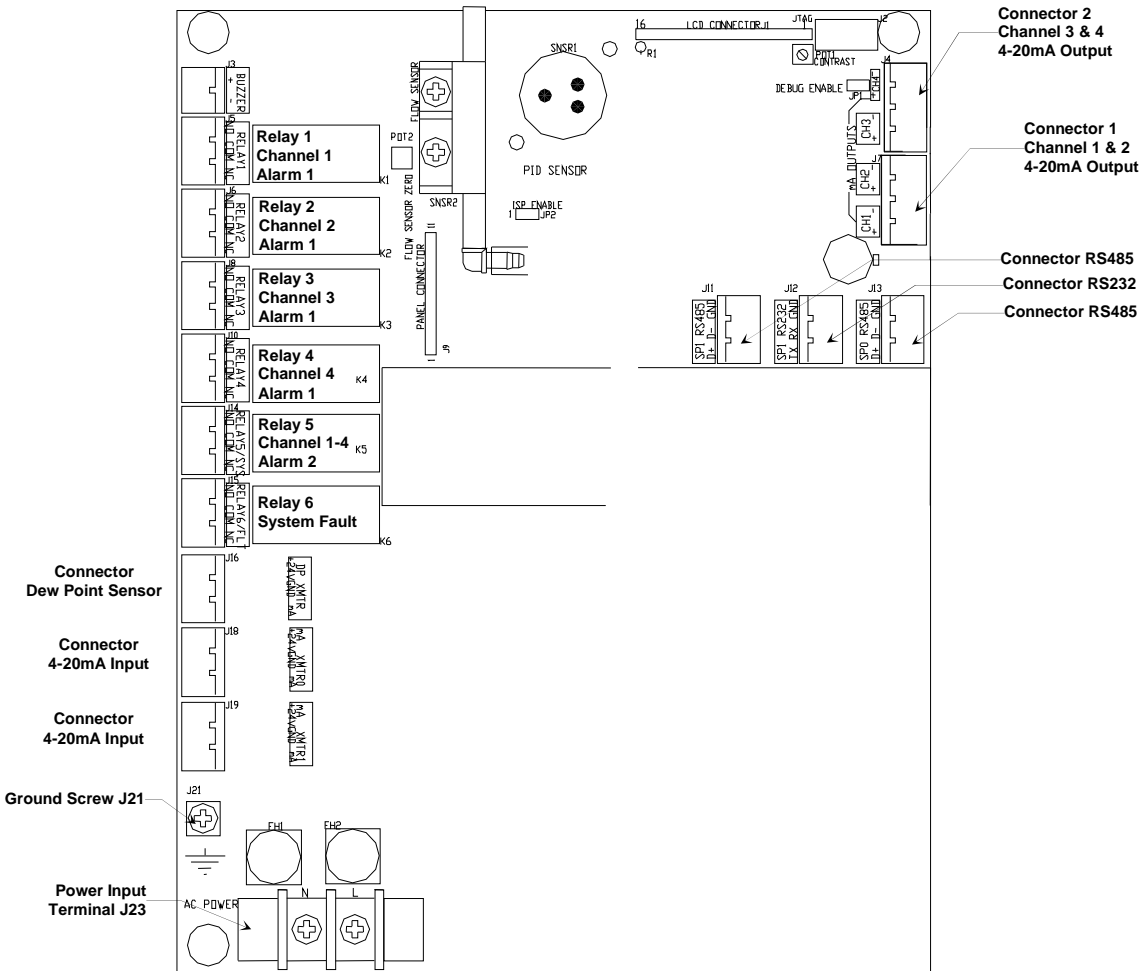


Figure 2A: Relay, Input and Output Terminals

3.4.1 Relay Contacts

Relay contacts are available for each alarm; these are SPDT, rated at 10Amp at 110VAC, and may be latching or non-latching as required by the application.

They are accessed on the terminals next to each relay see **Figure 2 & 2A**. The contact positions are noted on the circuit board next to each terminal.

Relays may also be configured as failsafe or non-failsafe. The default alarm relay configuration is for latching mode, and failsafe. They may be reconfigured in the maintenance menu. **See section 5.3.5 & 5.3.6**

The PC Board is labeled for the relays in their un-energized state. If the relay is configured for failsafe, then this is also the alarm condition state. Non-failsafe configured relays in the alarm state, are the reverse of the PC board labeling. Note that the Fault(FLT) relay cannot be set to operate in a Non-Failsafe mode. Please see the **Table 1** below:

Table 1 : Relay Failsafe Settings

Position	Failsafe-Alarm	Non-Failsafe-Alarm
J5 Relay 1 - NO	Normally Open	Normally Closed
J5 Relay 1 - COM	Common	Common
J5 Relay 1 - NC	Normally Closed	Normally Open
J6 Relay 2 - NO	Normally Open	Normally Closed
J6 Relay 2 - COM	Common	Common
J6 Relay 2 - NC	Normally Closed	Normally Open
J8 Relay 3 - NO	Normally Open	Normally Closed
J8 Relay 3 - COM	Common	Common
J8 Relay 3 - NC	Normally Closed	Normally Open
J10 Relay 4 - NO	Normally Open	Normally Closed
J10 Relay 4 - COM	Common	Common
J10 Relay 4 - NC	Normally Closed	Normally Open
J14 Relay 5 - NO	Normally Open	Normally Closed
J14 Relay 5 - COM	Common	Common
J14 Relay 5 - NC	Normally Closed	Normally Open
J15 Relay 6/FLT - NO	Normally Open	N/A
J15 Relay 6/FLT - COM	Common	N/A
J15 Relay 6/FLT - NC	Normally Closed	N/A

Relays can be linked to specific alarms. The table below shows the default relay links. They may be changed in the maintenance menu if required. See **Section 5.0**.

	Channel 1	Channel 2	Channel 3	Channel 4
Relay 1	Low Alarm			
Relay 2		Low Alarm		
Relay 3			Low Alarm	
Relay 4				Low Alarm
Relay 5	High Alarm	High Alarm	High Alarm	High Alarm

In addition, there is a fault relay, which changes state whenever the instrument is in a fault condition. The contact positions are noted on the circuit board next to each terminal. **See Figure 2A**. The coil of this relay is energized when the instrument is in the non-fault state; the contact conditions given on the circuit board next to the terminal, are for the non-energized state, which is identical to the fault state.

These relay contacts can be used to operate auxiliary alarms or other functions. It is recommended that power for auxiliary equipment be supplied from an independent power source, separate for the **MEDAIR 2200**. Place a hole in the enclosure for a wire exit, and use appropriate cable fittings. Be sure to note the location and depth of hardware inside the enclosure.

3.4.2 Optional 4-20mA Outputs

Isolated 4-20 mA outputs are available for data logging or other purposes. An output is supplied for each sensor supplied in a particular instrument, and can be added when a sensor is added in the field. These outputs are available on the Connector 1 for channels 1 & 2 and Connector 2 for channels 3 & 4.

4mA corresponds to a sensor reading at the bottom of the instrument range and 20mA corresponds to a full scale reading. Standard ranges are shown in **Table 2**.

Table 2: Sensor Output

Sensor	4mA	20mA
CO	0	50
Dew Point	-112 °F	68 °F
O2	0	30
CO ₂	0	5000
HC	0	100

Wiring requirements are the same as for the relays.


3.5 Installation Verification

All instruments are calibrated at the factory. You may, if a calibration kit is available, calibrate the any and all gas channels of the instrument 24 hours after installation to verify proper installation and instrument operation. See **Section 5.0, Maintenance**, for calibration instructions. Calibration is also recommended after the first month of operation. Subsequent calibrations should be performed every 3 months. The dew point sensor can not be calibrated in the field.

4.0 Operation

4.1 Normal Operation Condition

With the MEDIAIR 2200 installed as described in Section 3, and in clean air, the POWER green LED is on, the display is lit and the information on the display is as shown in Figure 4 Display, for the sensor(s) installed in the MEDIAIR 2200. The red alarm and fault LEDs are not lit.

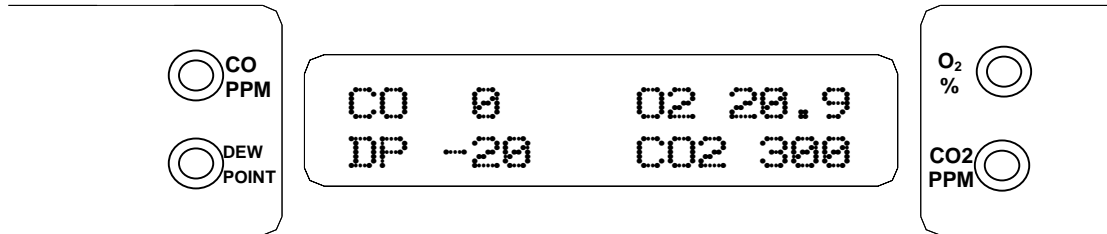

Example of display with CO(ch 1), Dew Point(ch 2), Oxygen(ch 3) and CO₂(ch 4) options installed

Figure 4: MEDIAIR 2200 Operational Display

4.2 Alarm Set Points

There are two alarm set points for each installed channel of the MEDIAIR 2200. The factory settings of these alarm set points are shown in Table 3.

Table 3: Typical Factory Alarm Set Points

Typical Channel #	Gas	Alarm 1, Flashing LED	Alarm 2, Steady LED
1	Carbon Monoxide	10 ppm	20 ppm
2	Dew point	39°Fahrenheit at 55PSIG	50°Fahrenheit at 55PSIG
3	Oxygen Deficiency	19.5 % by volume	23.5 % by volume
4	Carbon Dioxide	500 ppm	1000 ppm
4 or	Hydrocarbon	5 ppm	10 ppm

These alarm set points can be changed within limits; see the maintenance section of this manual for the procedure.

- If the CO concentration increases above that of the alarm set point, the associated red LED is lit, the associated relay changes state, and the audio alarm is activated.
- If the dew point increases above that of the alarm set point, the associated red LED is lit, the associated relay changes state, and the audio alarm is activated.
- If the oxygen content of the sample air decreases below the deficiency alarm set point, the associated red LED is lit, the associated relay changes state, and the audio alarm is activated.
- If the oxygen content of the sample air exceeds that of the abundance alarm set point, the associated red LED is lit, the audio alarm is activated, and both the oxygen alarm relay and the oxygen high alarm relay change state.
- The HC sensor can only detect and alarm to hydrocarbons with an Ionization Potential of less than 10.6 eV. See Appendix B.
- The HC sensor is broad range in nature and is unable to differentiate between different hydrocarbons.
- The Alarm 1 differential value is the delay of the MEDIAIR 2200 staying in alarm condition until after the measured reading has returned past the alarm point by the differential value. *Example:* If the alarm set point is 10 and the differential is 2, the MEDIAIR 2200 will go into alarm at 10 and stay in alarm until the reading has dropped below 8.

4.3 Alarm Latching or Differential Settings

An instrument is shipped with the alarms in the latching mode. The alarms may be independently configured in the non-latching mode or differential setting by use of the maintenance menu.

See *Section 5.3.3*, for setting alarm 1 and alarm 2.

Standard Setting

- IN THE LATCHING MODE: at the cessation of the condition which causes an alarm, the alarm indications do not cease, and the alarm relay contacts do not revert to the non-alarm state, until the **AUDIO DEFEAT / ALARM ACKNOWLEDGE** switch is pressed. An alarm can also be acknowledged by pressing the switch during the alarm condition; then at the cessation of the alarm condition, alarm indications cease and alarm relays revert to the non-alarm state. After an alarm is acknowledged, alarms in the latching configuration are re-armed to latch at the next alarm condition.
- IN THE NON-LATCHING MODE: at the cessation of the condition that causes an alarm, the alarm indications automatically cease, and the alarm relay contacts revert to the non-alarm state.

Differential Setting

- The Alarm 1 differential value is the delay of the **MEDAIR 2200** staying in alarm condition until after the measured reading has returned past the alarm point by the differential value. *Example:* If the alarm point is Λ 10 and the differential is 2, the **MEDAIR 2200** will go into alarm at 10 and stay in alarm until the reading has dropped below 8.

4.4 Audio Defeat

Pressing the **AUDIO DEFEAT / ALARM ACKNOWLEDGE** switch during an alarm temporarily silences the audio alarm. Relays and alarm LEDs continue to function, in the alarm state, during an alarm condition. As long as the alarm condition persists, the audio alarm will “chirp” every 20 seconds.

- If after 15 minutes the alarm condition continues the audio alarm will reactivate at full intensity.
- If any other alarm condition occurs while the audio alarm has been silenced it will force the audio alarm to reactivate immediately.

4.5 Display

In clean air a display is shown in **Figure 4**. This position of the display is termed the "**operational display**". As explained below, the display can be used to view other information by using the **OPTION** and **SELECT** switches.

Concentrations of CO and CO₂ are given in PPM (parts per million parts of air). Dew point is given in degrees Fahrenheit at 55 PSIG; *this can be changed to degrees Centigrade by pressing the **SELECT** switch*. Oxygen concentration is given in percent by volume.

When sample flow is reduced below a limit, the bottom line of the display flashes “Low Flow Alarm”.

4.6 Operational Menu

The operational menu allows the user to:

- View alarm set point concentration values
- View alarm ascending/descending trigger, latching and delay configurations
- Enter the maintenance menu with the proper Password.

The operational menu is accessed with the **OPTION** and **SELECT** switches. The operational menu flow chart is shown in **Figure 5**,

- Pressing the **OPTION** switch is indicated with a "O"
- Pressing the **SELECT** switch is indicated with a "S".

If the instrument is left at any location in the operational or maintenance menus, other than the operational display, with no action taken for a period of 45 seconds, it returns to the operational display.

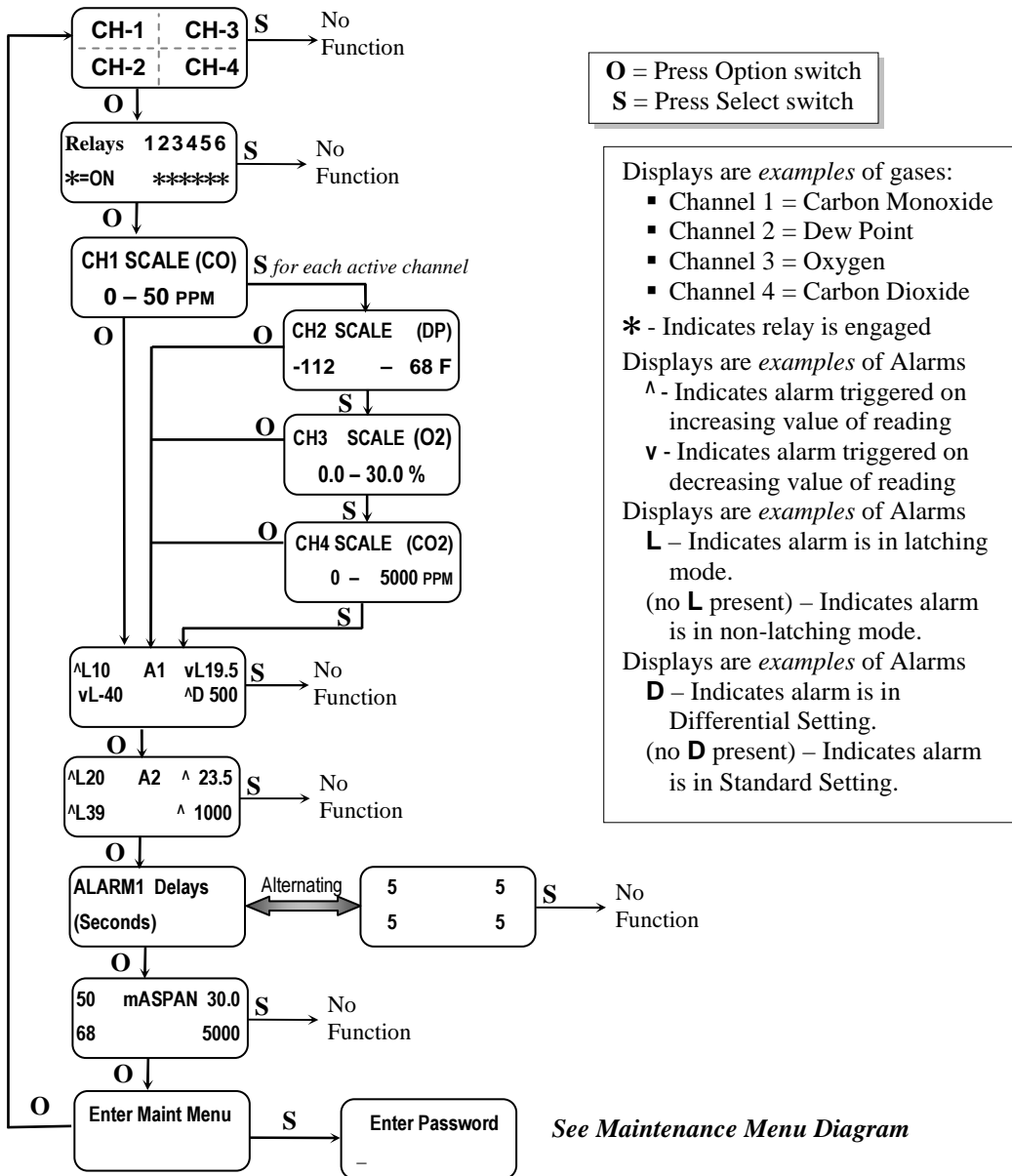


Figure 5: MEDIAIR 2200 Operation Menu Flow Chart

4.7 Fault Indications

4.7.1 Low Flow Indication

A flow sensor is used to furnish a low flow indication. When the sample air pressure drops below approximately 0.3 LPM, the fault light and audio alarm are activated, and the display flashes “Low Flow Alarm”.

4.7.2 Other Fault Indications

Other fault indications are associated with sensor zero and calibration activities, and are described in the maintenance **Section 5.0** of this manual.

4.8 Dew Point Sensor Response

It is a characteristic of the **MEDAIR 2200** that it takes more time to extract moisture from a sample by passing dry air through it, than it does to add moisture to a sample by passing moist air through it. Therefore, the time response of the instrument to a step change from moist to dry air is slower, than the response to a step change from dry to moist air.

It is the nature of most materials to absorb and release moisture at different rates. In general, it typically takes longer for a system to establish moisture equilibrium when going from a high to low humidity than it does to go from low to high. The **MEDAIR 2200** is no different. It may take up to 12 hours to establish moisture equilibrium when the instrument is first put on the air line if the Dew Point is less than 0°F. The sensor T90 response time is 10 seconds for a -40° to +50°F step change and 240 seconds for a +50° to -40°F step change. The delivery apparatus such as regulators, piping and tubing account for the additional response time of the instrument as a system.

4.9 Hydrocarbon Sensor Response

If a Hydrocarbon (HC) sensor is supplied with the **MEDAIR 2200** instrument, it designed to detect hydrocarbon gases and vapors with an ionization potential (IP) of 10.6 eV or less. Hydrocarbons with an IP of greater than 10.6 eV will NOT be detected. Please see Appendix B for a list of common gases and vapors and their respective IP rating. Unless otherwise noted Isobutylene is used as a calibration and reference gas.

5.0 Maintenance

The MEDIAIR 2200 requires periodic sensor calibration and replacement. Calibration of toxic gas and oxygen sensor should be performed immediately following installation, one month after installation and every 3 months thereafter. HC sensor should be calibrated on a monthly bases. Oxygen and CO sensor have an estimated lifetime of 1 – 3 years. The CO₂ sensor has an estimated lifetime of 3-5 years. Sensors should be replaced when they will not calibrate or shortly before the end of their estimated lifetime.

The dew point probe(sensor) can not be field calibrated. Dew point sensors should be exchanged with the factory on an annual bases.

5.1 Cleaning Instructions

CAUTION: Never spray a cleaning solution on the surfaces of the MEDIAIR 2200 devices.

Clean the exterior of the MEDIAIR 2200 enclosures with a mild soap solution on a clean, damp cloth. Do not soak the cloth with solution so that moisture drips onto, or lingers on, external surfaces.

Under no circumstances should organic solvents such as paint thinner be used to clean instrument surfaces.

5.2 Maintenance Menu

5.2.1 Accessing Maintenance Menu

The MEDIAIR 2200 maintenance menu is accessed by entering the proper password with the OPTION and SELECT switches. See Section 5.2.2 Figure 6 for full Maintenance Menu flow chart.


Entrance to the maintenance menu is guarded with a four-digit Password. The factory default setting of the password is 1270. When a valid numerical password is inserted, the user is allowed to enter the maintenance menu.

To enter the maintenance menu. Press the OPTION switch until "Enter Maint Menu" is displayed then press SELECT switch for the Enter Password menu. Enter the valid password as described below.


In the "Enter Maint Menu" position

- Press the SELECT switch "Enter Password █ 0" is displayed. Press SELECT switch once, to move cursor to next digit, this will be the first digit of the password.
- In the █000 position, the underline cursor is under the left digit.
- Press the OPTION switch to change the left digit; select the correct digit.
- Press the SELECT switch, which locks the digit in place and moves the cursor one digit to the right.

Continue this process until the four-digit password is complete. When a valid password is inserted in this manner, the display is transferred to the "Calibration" portion of the menu. If an invalid password is inserted you are returned to the Enter Maint Menu display.

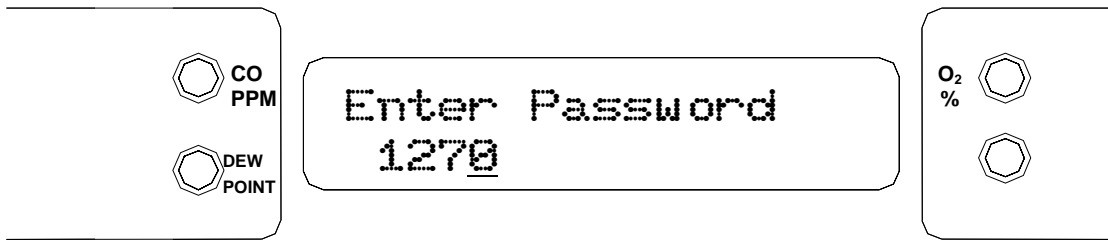

Example: Password Display (with factory installed password entered) and Flow Chart below.

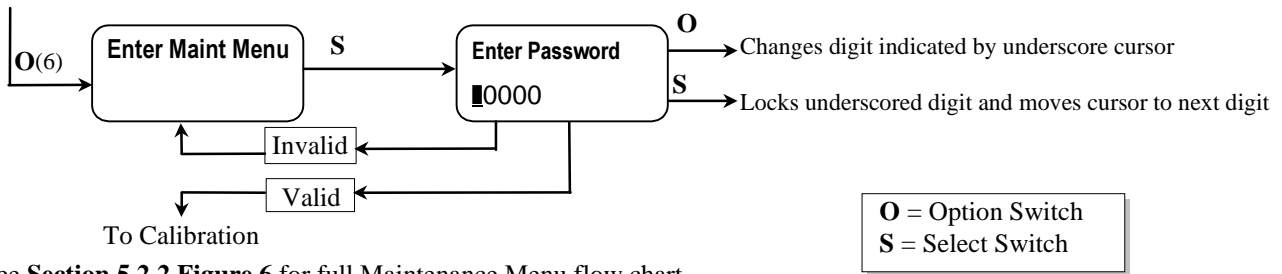

See Section 5.2.2 Figure 6 for full Maintenance Menu flow chart.

5.2.2 Maintenance Menu Flow Chart

The maintenance menu diagram is shown in **Figure 6 Maintenance Menu Flow Chart**. From the operational display, press the **OPTION** switch 6 times; "Enter MAINTENANCE Menu" is displayed.

O = Press Option
S = Press Select

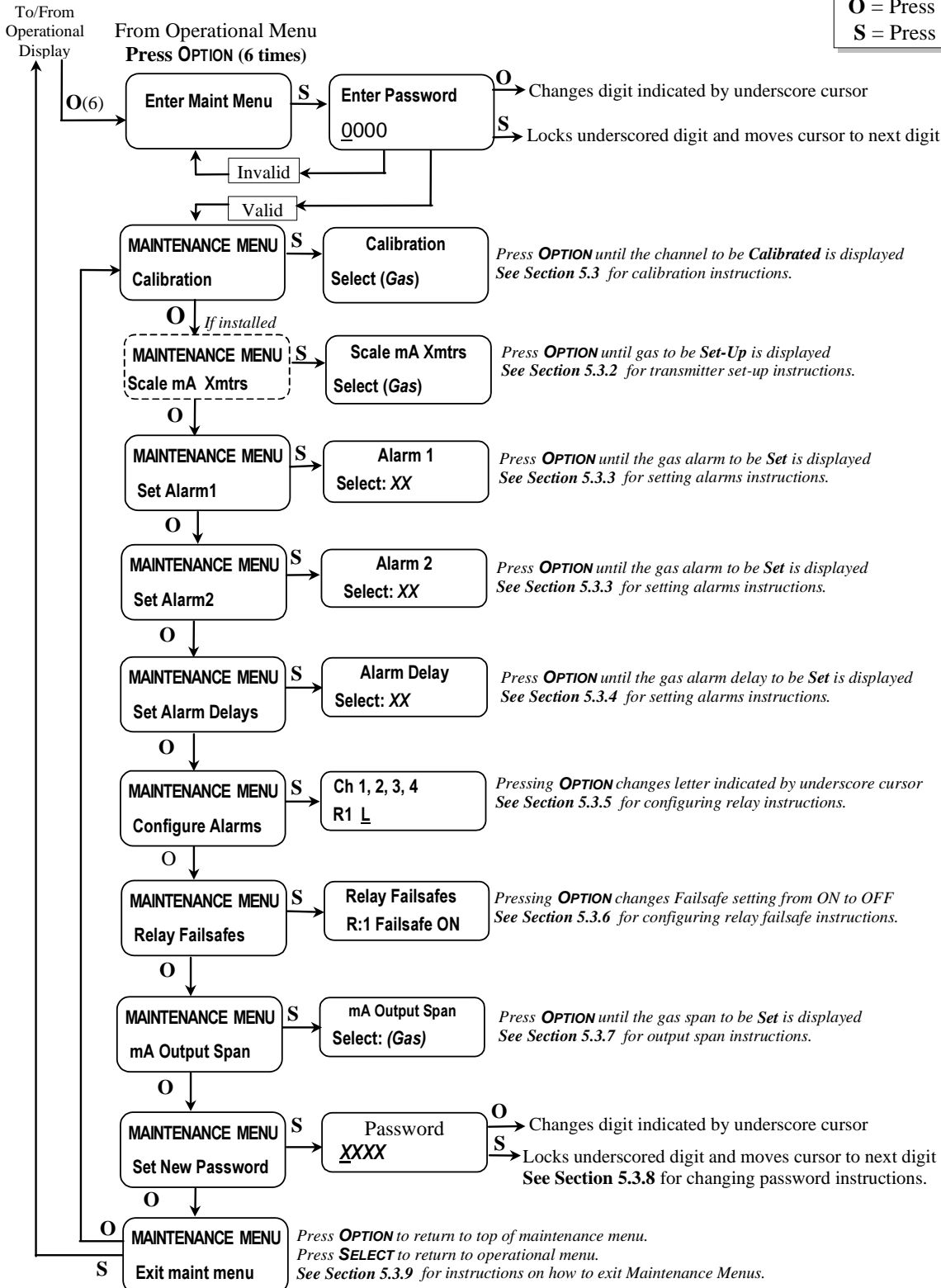


Figure 6: MEDIAIR 2200 Maintenance Menu Flow Chart.

5.3 Calibration for CO, O₂, HC and CO₂ (Gas Channels)

Calibration is the process of setting the instrument up to read accurately when exposed to a target gas. This is a two step process. A Low Calibration sets clean air reference point and the High Calibration function sets the sensitivity of the instrument.

Calibration equipment is available from **ENMET** Corporation to calibrate the **MEDIAIR 2200**. A list of needed material is in Section 7.0. A calibration adapter will have a fitting for the gas cylinder on one side, and a quick-disconnect to attach to the instrument on the other.

You may exit the calibration section, at any time, by *pressing and holding* the **OPTION** switch for 3 seconds, if entering calibration section by mistake or calibration gas is not available.

Wait 24 hours after initially supplying air and power to the **MEDIAIR 2200** sensor before initial calibration. It is not necessary to open the Front Panel to make adjustment. The calibration functions are operated through the **OPTION** and **SELECT** switches on the front panel.

After entering a valid password to maintenance menu, see **Section 5.2.1**, the calibration section is the first menu section; enter by pressing the **SELECT** switch.

Supply sensor with clean air for LowCal/ZeroCal setting and apply calibration gas for HiCal/SpanGas setting.

- Press the **SELECT** switch "Calibration Select XX" is displayed. XX = the gas to be calibrated
- Press the **OPTION** switch, if needed, to change to the gas to be calibrated.
- Press the **SELECT** switch, the gas & current reading are displayed in upper portion of display. The mV reading & "LowCal Q" is displayed in the lower portion of display. This is the LowCal setting, *usually zero*, clean air must be supplied to the sensor. This reading needs to be at or near zero. If it is not then a cylinder of clean 20.9 air should be used. See Figure 7 if this is required.
- Press the **SELECT** switch, that moves the cursor one digit to the right when the last digit is accepted the display will move to "HiCal xx" gas calibration. xx = the level of gas to be used for calibration. The mV reading is shown in the upper right hand corner of the display.
- Apply calibration gas to sensor. See **Figure 7**. After about 1 minute and mV reading has stabilized.
- Press the **SELECT** switch, that moves the cursor one digit to the right, when the last digit is accepted and the calibration is successful the display will momentarily show Cal OK then slope and off set readings, before returning to the Calibration Menu


Repeat above steps for each channel to be calibrated.

To continue on too next section Press the **OPTION** switch.

- Press **OPTION** switch until "Exit maint menu" appears and then press **SELECT** switch to return the instrument to the Operational Display

Example: Full Calibration Flow Chart, for CO

From Valid Password Entry

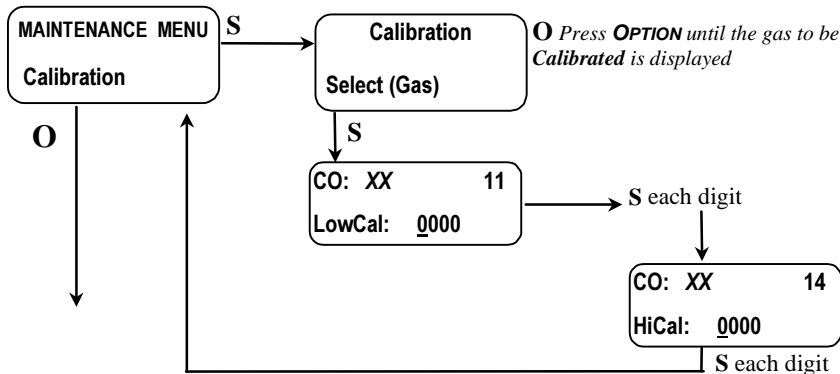

Default Calibration Points

Gas	LowCal	HiCal
CO	0	20
O ₂	N/A	20.9
CO ₂	0	1000
HC	0	10

O = Press Option
S = Press Select

NOTE: The dew point sensor/probe can not be calibrated in the field. **ENMET** offers a pre-calibrated dew point probe exchange program. Dew point probes can be exchanged with the factory, on an annual base. Contact **ENMET** customer service for cost and details.

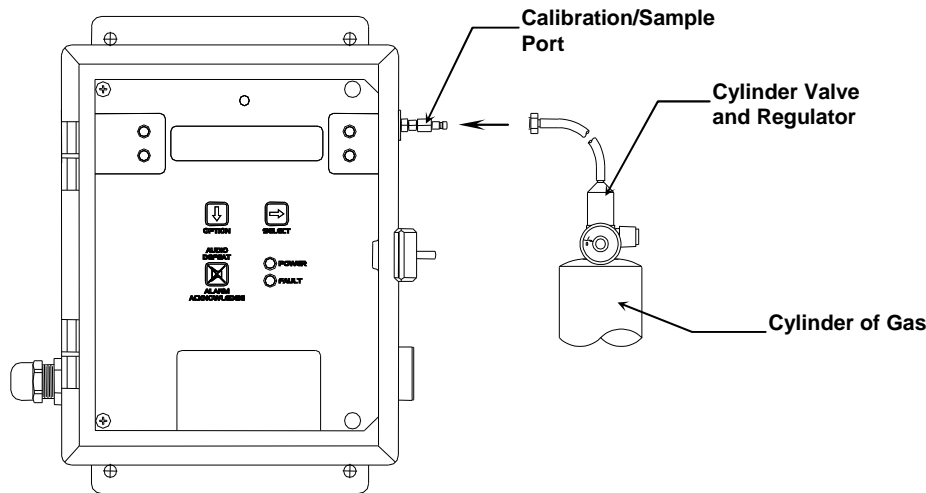


Figure 7: Connection of Calibration Gas Cylinder

5.3.1A Low Cal/ZeroCal Adjust

A Low Cal function should be performed only when the **MEDAIR 2200** sensor are exposed to clean uncontaminated air. Use a cylinder of 20.9% oxygen to provide a clean air reference if necessary. Attach the cylinder to the calibration adapter, attach the adapter to the instrument and allow gas to flow over the sensor for up to 4 minutes.

Enter the maintenance menu by repeatedly pressing **OPTION** switch, until the maintenance menu is displayed. See **Figure 6, MEDAIR 2200 Maintenance Menu** flow chart.

The first menu available is the Low Cal/ZeroCal.

Press the **SELECT** switch 4 times to perform a Low Cal.

- *If the Low Cal/ZeroCal is successful*, The display will change to Hi Cal/SpanGas. If you wish to Hi Cal/SpanGas the sensor apply calibration gas. **Proceed to gas calibration Section 5.3.1B**
If you wish to Exit the maintenance menu, Press and *hold* **OPTION** switch until the Maintenance Menu is displayed then release. Then press **OPTION** switch until “Exit maint menu” appears and then press **SELECT** switch to return the instrument to the Operational Display
- *If the Low Cal/ZeroCal is Not successful*, sensor is outside of safe parameters to Low Cal, a “SLP/Off Set err” will be indicated. Repeat Section 5.3.1 Low Cal/ZeroCal Adjust making sure to use a cylinder of 20.9% Oxygen.

5.3.1B High Cal/SpanGas Adjust

A High Cal/Span Gas should only be performed after a successful Low Cal/ZeroCal has been completed.

- Press the **SELECT** switch, that moves the cursor one digit to the right when the last digit is accepted the display will move to "HiCal *xx*" gas calibration. *xx* = the level of gas to be used for calibration. The mV reading is shown in the upper right hand corner of the display.
- Apply calibration gas to sensor. See **Figure 7**. After about 1 minute and mV reading has stabilized.
- Press the **SELECT** switch, that moves the cursor one digit to the right, when the last digit is accepted and the calibration is successful the display will momentarily show Cal OK then slope and off set readings, before returning to the Calibration Menu.
- *If High Cal is not successful*, sensor is outside of safe parameters to High Cal, a Cal Slop Err will be indicated. Verify calibration gas is correct and that the gauge on the regulator shows pressure in the cylinder.

Repeat above steps for each channel to be calibrated.

To continue on too next section Press the **OPTION** switch.

- Press **OPTION** switch until “Exit maint menu” appears and then press **SELECT** switch to return the instrument to the Operational Display

5.3.2 Set 4 –20mA Transmitter Scale


This section of the maintenance menu is installed when there are 4-20mA style sensors for dew point or other gases. This function is normally performed at the factory and is not usually required to be performed in the field unless a new transmitter is installed.

Consult the factory before entering this section of the maintenance menu.

After entering a valid password into maintenance menu, the Scale mA Xmtrs section is the second menu section, if it is installed, enter by pressing the SELECT switch

- Press the **SELECT** switch "mA Xmtr Scale: Select XX" is displayed. XX = the gas to be set up.
- Press the **OPTION** switch, if needed, to change to the gas to be set up.
- Press the **SELECT** switch, "Ch#: mAXmter: 4mA: 0000" is displayed
- Press the **SELECT** switch, that moves the cursor one digit to the right when the last digit is accepted the display move to the full Scale mA Xmtrs menu
- Press the **SELECT** switch, "Ch#: mAXmter: 20mA: 0000" is displayed
- Press the **SELECT** switch, that moves the cursor one digit to the right when the last digit is accepted the display will return to the Scale mA Xmtrs menu
- Repeat these steps for each 4 –20mA transmitter.
- Press **OPTION** switch until "Exit maint menu" appears and then press **SELECT** switch to return the instrument to the Operational Display

Example: Sensor/Transmitter Set Up Flow Chart

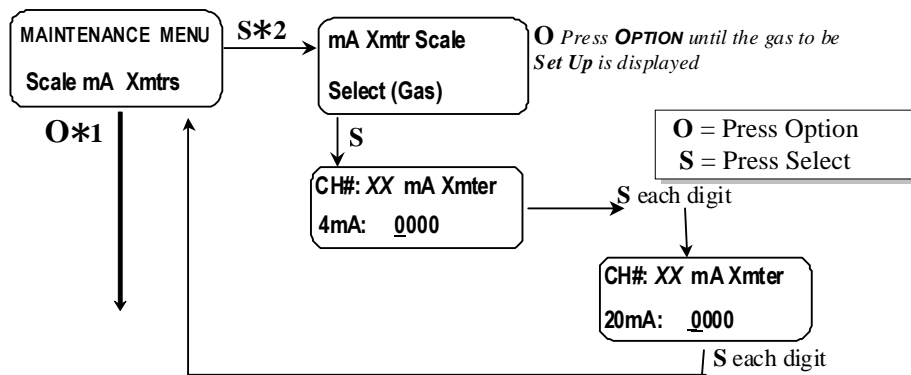

Example: Set up values

DP	-112F	68F

- *1 NOTE: Press **OPTION** do not press **SELECT**, Consult the factory prior to selecting **SELECT**.
- *2 NOTE: Consult the factory before entering or making any adjustments.

5.3.3 Set Alarm Points

Factory alarm set points are discussed in Section 4.2, See Table 1. To change the alarm points, you must enter the maintenance menu.

Entrance to the maintenance menu is guarded with a four-digit Password. The factory default setting of the password is 1270. When a valid numerical password is inserted, the user is allowed to enter the maintenance menu.

In the "Enter Maint Menu" position

- Press the **SELECT** switch "Enter Password █ 0" is displayed. Press **SELECT** switch once, to move cursor to next digit, this will be the first digit of the password.
- In the █000 position, the underline cursor is under the left digit.
- Press the **OPTION** switch to change the left digit; select the correct digit.
- Press the **SELECT** switch, which locks the digit in place and moves the cursor one digit to the right.

Continue this process until the four-digit password is complete. When a valid password is inserted in this manner, the display is transferred to the "Calibration" portion of the menu. If an invalid password is inserted you are returned to the Enter Maint Menu display.

After entering a valid password:

- Press the **OPTION** switch until; "Maintenance Menu Set Alarm1" appears on display.
- Press the **SELECT** switch, "ALARM1 Select: XX" is displayed. XX = the gas of alarm point to be changed.
- Press the **OPTION** switch until, desired gas is displayed.
- Press the **SELECT** switch; "ALARM 1 V" is displayed, with the flashing placeholder underscore cursor, under the left most character, **A** for ascending trigger point or **V** for descending trigger point indicator.
- Press the **OPTION** switch to toggle between **A** and **V**; select the correct indicator.
- Press the **SELECT** switch to lock in the correct indicator. "ALARM 1 **STD**" is displayed
- Press the **OPTION** switch to toggle between **STD** and **DIFF**; select the correct indicator.
- Press the **SELECT** switch to lock in the correct indicator.

If **STD** is selected, "ALARM 1 V**L** " is displayed.

▫The next character is the latching indicator **L** or **NOL** press the **OPTION** switch to toggle the latching mode.

▫The next character is the negative sign – press the **OPTION** switch to toggle the negative sign.

▫The next characters are the alarm 1 value, press the **OPTION** switch to select each digit of the value

When the last digit is accepted display returns to the "Set Alarm1" position.

If **DIFF** is selected, "ALARM 1 **A** **DIFF** 000" is displayed, to *set alarm 1 value*.

▫The next characters are the alarm 1 value, press the **OPTION** switch to select each digit of the value

▫Press the **SELECT** switch to lock in the correct character and move the cursor to the right.

▫ "ALARM 1 **DIFF** **BAND** 000" is displayed, to *set alarm 1 differential*.

▫Press the **OPTION** switch to select each digit of the value.

▫The next characters are the alarm 1 differential value, press the **OPTION** switch to select each digit of the value

▫Press the **SELECT** switch to lock in the correct character and move the cursor to the right.

When the last digit is accepted display returns to the "Set Alarm1" position.

Note: The Alarm 1 differential value is the delay of the **MEDAIR 2200** staying in alarm condition until after the measured reading has returned past the alarm point by the differential value. *Example:* If the alarm set point is **A** 10 and the differential is 2, the **MEDAIR 2200** will go into alarm at 10 and stay in alarm until the reading has dropped below 8.

- Repeat for each sensor alarm 1 to be changed.
- Press the **OPTION** switch to move to alarm 2, "Set ALARM2" is displayed.
- Repeat as for alarm 1 using the **STD** section.
- Press **OPTION** switch until "Exit maint menu" appears, then press **SELECT** switch to return the instrument to the Operational Display

5.3.4 Set Alarm Delay

The alarms may be set to delay by 1 second increments, up to 255 seconds. Alarm delays are factory set to 5 seconds.

To change an alarm delay, you must enter the maintenance menu. Press the **OPTION** switch until "Enter Maint Menu" is displayed then press **SELECT** switch for the Enter Password menu. Enter the valid password as described in **Section 5.2.1**. See **Table 4** below for factory set delays. A space is provided to record changes.

After entering a valid password:

- Press the **OPTION** switch until; "Maintenance Menu Set Alarm Delay" appears on display.
- Press the **SELECT** switch, "ALARM Delay Select: XX" is displayed. XX = the gas alarm to be changed.
- Press the **OPTION** switch until, desired gas is displayed.
- Press the **SELECT** switch; "ALARM Delay = 0000" is displayed, with the underscore cursor under the left digit.
- Press the **OPTION** switch to change the left digit; select the correct digit.
- Press the **SELECT** switch to lock in the correct digit and move the cursor one digit to the right. When the last digit is accepted display returns to the "Set Alarm Delay" position.
- Repeat for each sensor alarm delay to be changed.
- Press **OPTION** switch until "Exit maint menu" appears and then press **SELECT** switch to return the instrument to the Operational Display

Example: Set Alarm Delay Flow Chart

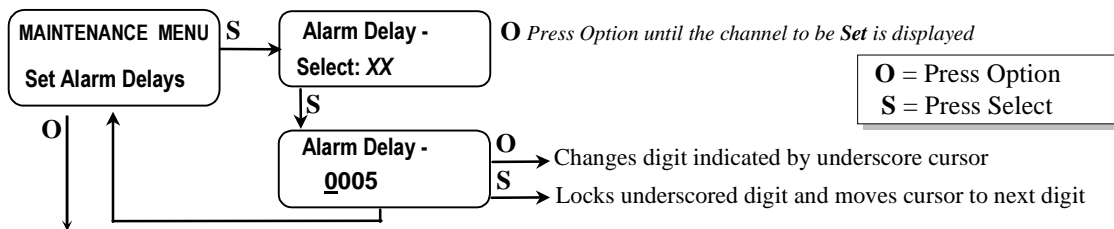


Table 4: Factory Set Gas alarms Delay

Gas	Delay	
CO	5 sec	
DP	5 sec	
O ₂	5 sec	
CO ₂	5 sec	

5.3.5 Relay Configuration

To change a relay configuration you must enter the maintenance menu. Press the **OPTION** switch until “Enter Maint Menu” is displayed then press **SELECT** switch for the Enter Password menu. Enter the valid password as described below.

In the "Enter Maint Menu" position


- Press the **SELECT** switch "Enter Password █ 0" is displayed. Press **SELECT** switch once, to move cursor to next digit, this will be the first digit of the password.
- In the █000 position, the underline cursor is under the left digit.
- Press the **OPTION** switch to change the left digit; select the correct digit.
- Press the **SELECT** switch, which locks the digit in place and moves the cursor one digit to the right.

Continue this process until the four-digit password is complete. When a valid password is inserted in this manner, the display is transferred to the "Calibration" portion of the menu. If an invalid password is inserted you are returned to the Enter Maint Menu display.

After entering a valid password:

- Press the **OPTION** switch until “Configure Alarms” is displayed
- Press the **SELECT** switch to enter the Configure Alarms menu
- Press the **OPTION** switch to set relay configuration as needed, see below for indications
L = Low Alarm, **H** = High Alarm, **B** = Both Alarms, **█** = No Relay linked to channel
- Press the **SELECT** switch to lock setting and move to next, channel and relay
- Press **OPTION** switch until “Exit maint menu” appears and then press **SELECT** switch to return the instrument to the Operational Display

Example: Set Relay Configuration Flow Chart

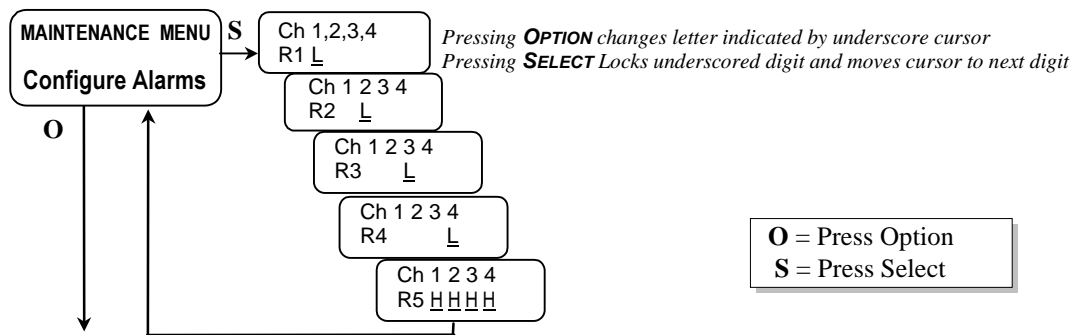

The table below shows the default relay links.

	Channel 1	Channel 2	Channel 3	Channel 4
Relay 1	Low Alarm			
Relay 2		Low Alarm		
Relay 3			Low Alarm	
Relay 4				Low Alarm
Relay 5	High Alarm	High Alarm	High Alarm	High Alarm

Relays can be linked to specific alarms.

NOTE: Each operating channel must be linked to at least 1 relay.

5.3.6 Failsafe Configuration

To change a relay failsafe configuration you must enter the maintenance menu. Press the **OPTION** switch until "Enter Maint Menu" is displayed then press **SELECT** switch for the Enter Password menu. Enter the valid password as described below.

In the "Enter Maint Menu" position


- Press the **SELECT** switch "Enter Password █ 0" is displayed. Press **SELECT** switch once, to move cursor to next digit, this will be the first digit of the password.
- In the █000 position, the underline cursor is under the left digit.
- Press the **OPTION** switch to change the left digit; select the correct digit.
- Press the switch, which locks the digit in place and moves the cursor one digit to the right.

Continue this process until the four-digit password is complete. When a valid password is inserted in this manner, the display is transferred to the "Calibration" portion of the menu. If an invalid password is inserted you are returned to the Enter Maint Menu display.

After entering a valid password:

- Press the **OPTION** switch until "Relay Failsafes" is displayed
- Press the **SELECT** switch to indicate relay to be set.
- Press the **OPTION** switch to set relay indicated, On or Off as appropriate.
- Press the **SELECT** switch to cycle through each of the 5 relays, return to "Maintenance Menu Relay Failsafes"
- Press **OPTION** switch until "Exit maint menu" appears and then press **SELECT** switch to return the instrument to the Operational Display

Example: Set Relay Failsafe Configuration Flow Chart

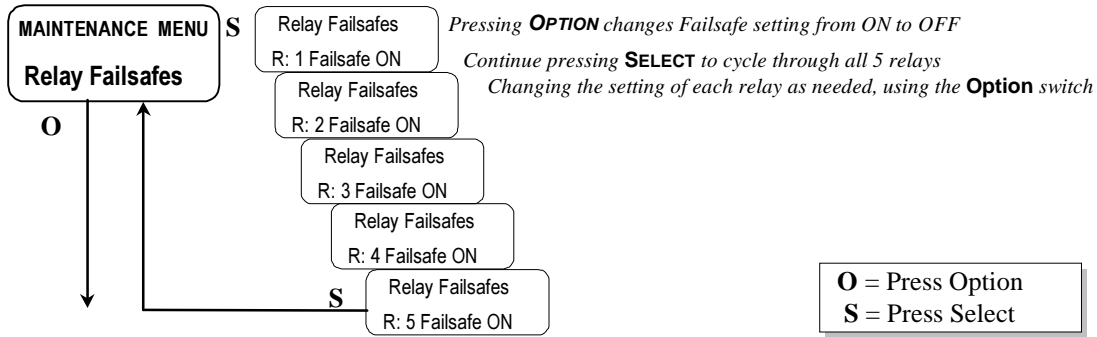


5.3.7 Set Output Span Range

To change 4-20 mA output range. This range is set at the factory and should not be changed, contact **ENMET** for information.

- Press the **OPTION** switch to continue to next section of maintenance menu.
- Press **OPTION** switch until "Exit maint menu" appears and then press **SELECT** switch to return the instrument to the Operational Display

Example: Set Output Span Flow Chart

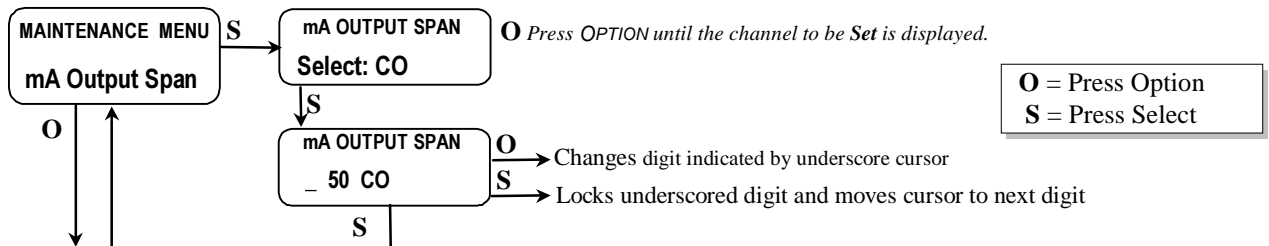

5.3.8 Set New Password

To change the password, you must enter the maintenance menu. Press the **OPTION** switch until "Enter Maint Menu" is displayed then press **SELECT** switch for the Enter Password menu. Enter the valid password as described in Section 5.2.1.

To set a new password, after inserting a valid password,

- Press the **OPTION** switch until; "Set New Password" is displayed.
- Press the **SELECT** switch; "Password █1270" is displayed, with the underscore cursor under the left digit.
- Use the **OPTION** switch to change the left digit, when the desired digit is displayed.
- Press the **SELECT** switch to lock the digit in place and move the cursor one digit to the right.


When all four digits of the new password have been selected, "Set New Password" is displayed.

Record the new password; without it, the maintenance menu cannot be reentered once you exit the Maintenance Menu. If the password is lost, call **ENMET** customer service personnel.

From the "Password XXXX" position,

- Press the **SELECT** switch to return to Set New Password section.
- Press the **OPTION** switch; to continue to "exit MAINTENANCE Menu"

Example: Set Password Flow Chart

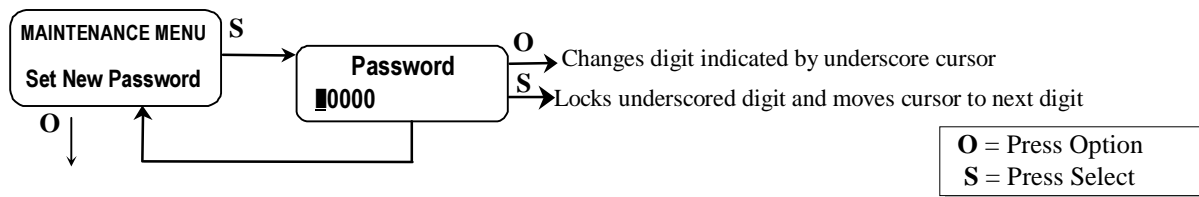


5.3.9 Exit Maintenance Menu

From the "exit MAINTENANCE Menu" position

- Press the **SELECT** switch to resume the operational display.
- Press the **OPTION** switch to reenter the maintenance menu at the "Calibration" position.

Example: Exit Maintenance Menu Flow Chart

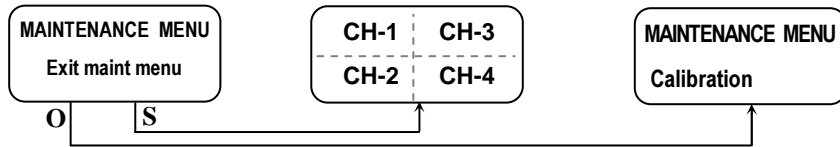

After entering a valid password to maintenance menu, see **Section 5.2.1**, the calibration section is the first menu section; enter by pressing the **SELECT** switch.

Supply sensor with clean air for LowCal/ZeroCal setting and apply calibration gas for HiCal/SpanGas setting.


- Press the **SELECT** switch "Calibration Select XX" is displayed. XX = the gas to be calibrated
- Press and *Hold* the **OPTION** switch, until the letter F appears in the upper right hand corner of the display. The F indicates that the instrument is in the Factory Calibration Mode
- Press the **OPTION** switch, if needed, to change to the gas to be calibrated.
- Press the **SELECT** switch, the gas & current reading are displayed in upper portion of display. The mV reading & "LowCal 0" is displayed in the lower portion of display. This is the LowCal setting, *usually zero*, clean air must be supplied to the sensor. This reading needs to be at or near zero. If it is not then a cylinder of clean 20.9 air should be used. See Figure 7 if this is required.
- Press the **SELECT** switch, that moves the cursor one digit to the right when the last digit is accepted the display will move to "HiCal xx" gas calibration. xx = the level of gas to be used for calibration. The mV reading is shown in the upper right hand corner of the display.
- Apply calibration gas to sensor. See **Figure 7**. After about 1 minute and mV reading has stabilized.
- Press the **SELECT** switch, that moves the cursor one digit to the right, when the last digit is accepted and the calibration is successful the display will momentarily show Cal OK then slope and off set readings, before returning to the Calibration Menu

Repeat above steps for each channel to be calibrated.

To continue on too next section Press the **OPTION** switch.

- Press **OPTION** switch until "Exit maint menu" appears and then press **SELECT** switch to return the instrument to the Operational Display

Example: Full Calibration Flow Chart, for CO
From Valid Password Entry

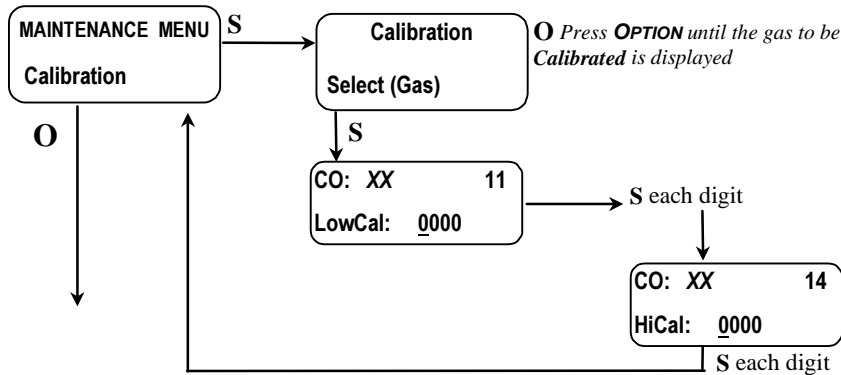

Default Calibration Points

Gas	LowCal	HiCal
CO	0	20
O2	N/A	20.9
CO ₂	0	1000
HC	0	10

O = Press Option
S = Press Select

NOTE: The dew point sensor/probe can not be calibrated in the field. **ENMET** offers a pre-calibrated dew point probe exchange program. Dew point probes can be exchanged with the factory, on an annual base. Contact **ENMET** customer service for cost and details.

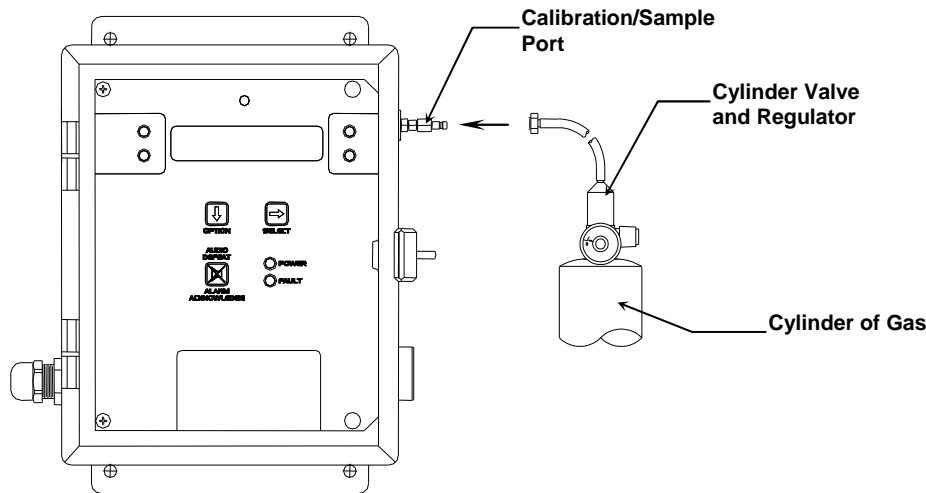


Figure 7: Connection of Calibration Gas Cylinder

5.4.3A Low Cal/ZeroCal Adjust

A Low Cal function should be performed only when the **MEDAIR 2200** sensor are exposed to clean uncontaminated air. Use a cylinder of 20.9% oxygen to provide a clean air reference if necessary. Attach the cylinder to the calibration adapter, attach the adapter to the instrument and allow gas to flow over the sensor for up to 4 minutes.

Enter the maintenance menu by repeatedly pressing **OPTION** switch, until the maintenance menu is displayed. See **Figure 6, MEDAIR 2200 Maintenance Menu** flow chart.

The first menu available is the Low Cal/ZeroCal.

Press the **SELECT** switch 4 times to perform a Low Cal.

- *If the Low Cal/ZeroCal is successful*, The display will change to Hi Cal/SpanGas. If you wish to Hi Cal/SpanGas the sensor apply calibration gas. **Proceed to gas calibration Section 5.3.1B**
If you wish to Exit the maintenance menu, Press and *hold* **OPTION** switch until the Maintenance Menu is displayed then release. Then press **OPTION** switch until “Exit maint menu” appears and then press **SELECT** switch to return the instrument to the Operational Display
- *If the Low Cal/ZeroCal is Not successful*, sensor is outside of safe parameters to Low Cal, a “SLP/Off Set err” will be indicated. Repeat Section 5.3.1 Low Cal/ZeroCal Adjust making sure to use a cylinder of 20.9% Oxygen.

5.4.3B High Cal/SpanGas Adjust

A High Cal/Span Gas should only be preformed after a successful Low Cal/ZeroCal has been completed.

- Press the **SELECT** switch, that moves the cursor one digit to the right when the last digit is accepted the display will move to "HiCal *xx*" gas calibration. *xx* = the level of gas to be used for calibration. The mV reading is shown in the upper right hand corner of the display.
- Apply calibration gas to sensor. See **Figure 7**. After about 1 minute and mV reading has stabilized.
- Press the **SELECT** switch, that moves the cursor one digit to the right, when the last digit is accepted and the calibration is successful the display will momentarily show Cal OK then slope and off set readings, before returning to the Calibration Menu.
- *If High Cal is not successful*, sensor is outside of safe parameters to High Cal, a Cal Slop Err will be indicated. Verify calibration gas is correct and that the gauge on the regulator shows pressure in the cylinder.

Repeat above steps for each channel to be calibrated.

To continue on too next section Press the **OPTION** switch.

- Press **OPTION** switch until “Exit maint menu” appears and then press **SELECT** switch to return the instrument to the Operational Display

5.5 Dew Point Sensor

Unlike the CO and O₂ sensors, the dew point sensor cannot be field calibrated. To assure correct performance, the dew point sensor should be replaced annually. To minimize instrument downtime, take advantage of the dew point sensor exchange program available through **ENMET**, in which an old dew point sensor can be exchanged for a newly calibrated sensor. Call **ENMET** customer service personnel for details.

5.6 Flow Control Orifice / Pre-filter

A 0.005-inch diameter orifice is used to set the flow rate and to drop the air pressure. A pre-filter is in line to help prevent clogging. In well-maintained medical air systems, this orifice should not clog. However, if difficulty is experienced in maintaining flow rate with assured inlet pressure, remove air pressure from the equipment and examine this orifice and pre-filter; replace it if necessary.

Orifice location will depend on instrument sensor configuration. Orifice and pre-filter are located at the output point of the dew point probe manifold or on the inlet line to the gas sensor manifold.

6.0 Technical Data and Specifications

Electrical Power	15 Amp fused branch circuit			
	100-240 VAC			
	0.9 A			
	50/60 Hz			
	Board Mounted Fuse FH2, 0.630A, 5 x 20mm			
Storage and Transport	Temperature:		-20° to +60°C (-4° to +140°F)	
	<i>preferred</i>		0° to +20°C (32° to 68°F)	
	Relative Humidity		0 - 99% RH, non-condensing	
	Atmospheric Pressure		20 to 36 inHg (68 to 133 kPa)	
Operation	Temperature:		0° to +40°C (32° to +104°F)	
	Relative Humidity		0 - 99% RH, non-condensing	
	Atmospheric Pressure		20 to 36 inHg (68 to 133 kPa)	
	Air Line Pressure		55 PSI (± 5 PSI)	
Mechanical	Dimensions:		11 x 9 x 6 inches (4.3 x 3.5 x 2.4 cm)	
	Weight:		8 lbs (3.6 kg)	
	Material:		Engineered thermoplastic with hinged front cover	
	Strain relief:		5-12 mm OD	
Outputs	Relays:		SPDT Resistive Load Inductive Load 10A at 110 VAC 7.5A at 110 VAC 10A at 30 VDC 5A at 30 VDC	
	Analog:		4-20 mA x 3	
	Digital:		RS-232 – Modbus RS-485 – Modbus	
	Audio:		95 db at 2 ft	
Sensors	Type	Range	Response Time	Life
	CO	0 – 50 ppm	T ₉₀ = 30 seconds	1 – 3 years
	Dew Point	-112 - +68°F	T ₉₀ = 10 seconds for -40°F to 50°F step change	5+ years
	O ₂	0 – 30%	T ₉₀ = 15 seconds	1 – 2 years
	CO ₂	0 – 5000 ppm	T ₉₀ = 30 seconds	3 – 5 years
	HC	0 – 100 ppm	T ₉₀ = 30 seconds	1 – 2 years

NOTE: All specifications stated in this manual may change without notice.

7.0 Replacement Part Numbers

7.1 ENMET part numbers for sensors and replacement parts:

Part number	Description
03009-005	Dew Point Probe (New Probe)
03009-006	Dew Point Probe (Exchange: Pre-calibrated Probe)
03053-000	Sensor, CO ₂
67025-1114	Sensor, Oxygen
67025-1200	Sensor, CO
03028-000	Sensor, PID, 10.6 eV
03028-005	Sensor, Replacement lamp, PID, 10.6 eV
73540-701	Orifice
73583-700	Pre-filter, orifice
64002-630	Fuse, 0.630 Amp 5x20mm
06008-004	Sensor Gasket
65057-011	Terminal plug, 3 position
65057-012	Terminal plug, 4 position
65057-010	Terminal plug, 2 position

7.2 ENMET part numbers for Calibration equipment:

Part number	Description
03219-020	Gas Cylinder, 20 ppm CO in air, (17ℓ cylinders)
03296-209	Gas Cylinder, 20.9% oxygen in nitrogen, (17ℓ cylinders)
03700-500	Calibration Adapter, CO, O ₂ (17ℓ cylinders)
03223-1000	Gas Cylinder, 1000 ppm CO ₂ in air, (103ℓ cylinders)
03700-501	Calibration Adapter, CO ₂ (34, 58, 103ℓ cylinders)
03290-010	Gas Cylinder, 10 ppm Isobutylene, (17ℓ cylinders)

8.0 WARRANTY

ENMET warrants new instruments to be free from defects in workmanship and material under normal use for a period of one year from date of shipment from **ENMET**. The warranty covers both parts and labor excluding instrument calibration and expendable parts such as calibration gas, filters, batteries, etc... Equipment believed to be defective should be returned to **ENMET** within the warranty period (transportation prepaid) for inspection. If the evaluation by **ENMET** confirms that the product is defective, it will be repaired or replaced at no charge, within the stated limitations, and returned prepaid to any location in the United States by the most economical means, e.g. Surface UPS/FedEx Ground. If an expedient means of transportation is requested during the warranty period, the customer is responsible for the difference between the most economical means and the expedient mode. **ENMET** shall not be liable for any loss or damage caused by the improper use of the product. The purchaser indemnifies and saves harmless the company with respect to any loss or damages that may arise through the use by the purchaser or others of this equipment.

This warranty is expressly given in lieu of all other warranties, either expressed or implied, including that of merchantability, and all other obligations or liabilities of **ENMET** that may arise in connection with this equipment. **ENMET** neither assumes nor authorizes any representative or other person to assume for it any obligation or liability other than that, which is set forth herein.

NOTE: When returning an instrument to the factory for service:

- Be sure to include paperwork.
- A purchase order, return address and telephone number will assist in the expedient repair and return of your unit.
- Include any specific instructions.
- For warranty service, include date of purchase
- If you require an estimate, please contact **ENMET** Corporation.

There is Return for Repair Instructions and Form on the last pages of this manual. This form can be copied or used as needed.

Manual Part Number:

80002-042

October 2006

MCN-344, 03/23/06
MCN-345, 04/12/06
MCN-346, 04/21/06
MCN-351, 08/21/06
MCN-353, 11/03/06
MCN-363, 02/28/07
MCN-369, 06/18/07
MCN-391, 11/15/07
MCN-395, 01/09/08
MCN-398, 03/10/08
MCN-422, 06/16/09
MCN-439, 08/20/10

Appendix A: CO Characteristics

The Characteristics and Effects of Carbon Monoxide

Carbon monoxide is a colorless odorless toxic gas generated by incomplete combustion of a hydrocarbon fuel in air. It may be present where internal combustion engines, furnaces, boilers, and other combustion devices are present. It is toxic when inhaled because of its great affinity to hemoglobin, the oxygen carriers in the red cells of the blood. CO replaces the oxygen normally carried by the hemoglobin, and thus inhibits the delivery of oxygen throughout the body; the victim suffers from oxygen deficiency, and may die from asphyxiation. The symptoms and degree of danger resulting from exposure to CO depend upon the concentration of the gas and the length of exposure; this is shown in **Figure 10**. The **MEDAIR 2200** instrument is employed to warn the user of the presence of CO, and to facilitate the assessment of the degree of danger that he or she is exposed to.

Based upon knowledge of the effects of CO, the Occupational Safety and Health Authority (OSHA) has set limits on exposure to CO in the workplace. These are 35 ppm (parts CO per million parts air) as an time weighted average for an eight hour day, and a maximum exposure of 200 ppm. For compressed air line applications, OSHA requires Grade D breathing air supplied, using a Compressed Gas Association (CGA) definition (G-7.1). Depending on interpretation of the OSHA respiratory standard, 10 ppm and 20 ppm CO have been used as maximum limits and standard instrument alarm points.

If a CO sensor is installed, the **MEDAIR 2200** has two preset alarm set points, at 10 ppm and 20 ppm CO, which are adjustable, but cannot be set below 5 ppm or above 50 ppm.

The curves below are for percent Carboxyhemaglobin with 50% being the top curve, 5% the bottom. %COHb is a measure of the amount of hemoglobin occupied by CO rather than oxygen. CO effects upon children, adults engaging in physical activity, and smokers, are more pronounced.

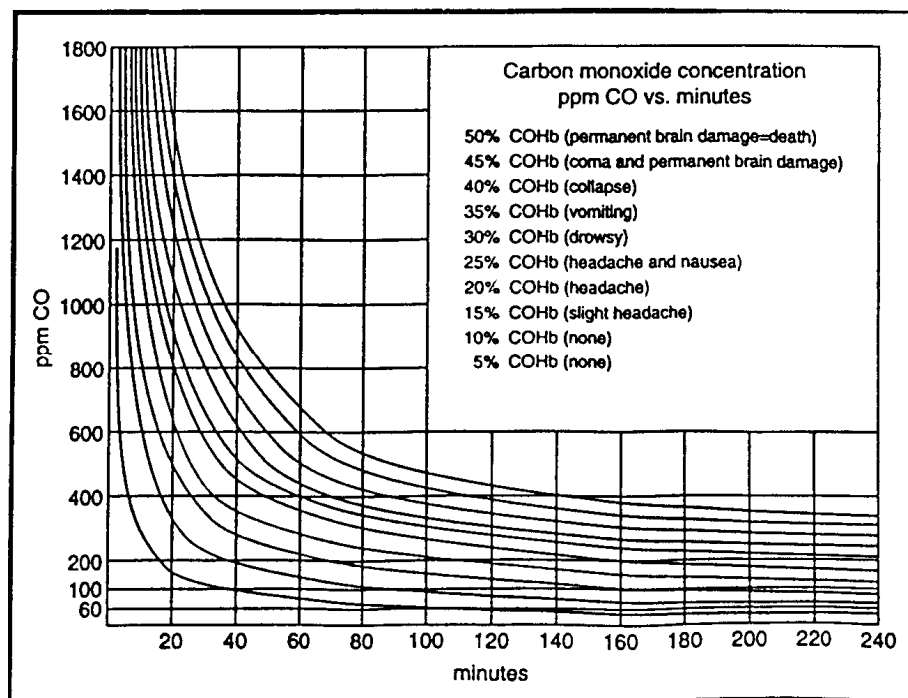


Figure 10: Carbon Monoxide Concentration

Appendix B: Gas Ionization Potentials

Chemical Name	IP (eV)
A	
2-Amino pyridine	8.00
Acetaldehyde	10.21
Acetamide	9.77
Acetic acid	10.69
Acetic anhydride	10.00
Acetone	9.69
Acetonitrile	12.20
Acetophenone	9.27
Acetyl bromide	10.55
Acetyl chloride	11.02
Acetylene	11.41
Acrolein	10.10
Acrylamide	9.50
Acrylonitrile	10.91
Allyl alcohol	9.67
Allyl chloride	9.90
Ammonia	10.20
Aniline	7.70
Anisidine	7.44
Anisole	8.22
Arsine	9.89
B	
1,3-Butadiene (butadiene)	9.07
1-Bromo-2-chloroethane	10.63
1-Bromo-2-methylpropane	10.09
1-Bromo-4-fluorobenzene	8.99
1-Bromobutane	10.13
1-Bromopentane	10.10
1-Bromopropane	10.18
1-Bromopropene	9.30
1-Butanethiol	9.14
1-Butene	9.58
1-Butyne	10.18
2,3-Butadiene	9.23
2-Bromo-2-methylpropane	9.89
2-Bromobutane	9.98
2-Bromopropane	10.08
2-Bromothiophene	8.63
2-Butanone (MEK)	9.54
3-Bromopropene	9.70
3-Butene nitrile	10.39
Benzaldehyde	9.53
Benzene	9.25
Benzenethiol	8.33
Benzonitrile	9.71
Benzotrifluoride	9.68
Biphenyl	8.27
Boron oxide	13.50
Boron trifluoride	15.56
Bromine	10.54
Bromobenzene	8.98
Bromochloromethane	10.77
Bromoform	10.48
Butane	10.63
Butyl mercaptan	9.15
cis-2-Butene	9.13
m-Bromotoluene	8.81
n-Butyl acetate	10.01
n-Butyl alcohol	10.04
n-Butyl amine	8.71
n-Butyl benzene	8.69
n-Butyl formate	10.50
n-Butyraldehyde	9.86
n-Butyric acid	10.16
n-Butyronitrile	11.67
o-Bromotoluene	8.79

Chemical Name	IP (eV)
p-Bromotoluene	8.67
p-tert-Butyltoluene	8.28
s-Butyl amine	8.70
s-Butyl benzene	8.68
sec-Butyl acetate	9.91
t-Butyl amine	8.64
t-Butyl benzene	8.68
trans-2-Butene	9.13
C	
1-Chloro-2-methylpropane	10.66
1-Chloro-3-fluorobenzene	9.21
1-Chlorobutane	10.67
1-Chloropropane	10.82
2-Chloro-2-methylpropane	10.61
2-Chlorobutane	10.65
2-Chloropropane	10.78
2-Chlorothiophene	8.68
3-Chloropropene	10.04
Camphor	8.76
Carbon dioxide	13.79
Carbon disulfide	10.07
Carbon monoxide	14.01
Carbon tetrachloride	11.47
Chlorine	11.48
Chlorine dioxide	10.36
Chlorine trifluoride	12.65
Chloroacetaldehyde	10.61
α -Chloroacetophenone	9.44
Chlorobenzene	9.07
Chlorobromomethane	10.77
Chlorofluoromethane (Freon 22)	12.45
Chloroform	11.37
Chlorotrifluoromethane (Freon 13)	12.91
Chrysene	7.59
Cresol	8.14
Crotonaldehyde	9.73
Cumene (isopropyl benzene)	8.75
Cyanogen	13.80
Cyclohexane	9.80
Cyclohexanol	9.75
Cyclohexanone	9.14
Cyclohexene	8.95
Cyclo-octatetraene	7.99
Cyclopentadiene	8.56
Cyclopentane	10.53
Cyclopentanone	9.26
Cyclopentene	9.01
Cyclopropane	10.06
m-Chlorotoluene	8.83
o-Chlorotoluene	8.83
p-Chlorotoluene	8.70
D	
1,1-Dibromoethane	10.19
1,1-Dichloroethane	11.12
1,1-Dimethoxyethane	9.65
1,1-Dimethylhydrazine	7.28
1,2-Dibromoethane	9.45
1,2-Dichloro-1,1,2,2-tetrafluoroethane (Freon 114)	12.20
1,2-Dichloroethane	11.12
1,2-Dichloropropane	10.87
1,3-Dibromopropane	10.07
1,3-Dichloropropane	10.85
2,2-Dimethyl butane	10.06
2,2-Dimethyl propane	10.35
2,3-Dichloropropene	9.82
2,3-Dimethyl butane	10.02
3,3-Dimethyl butanone	9.17
cis-Dichloroethene	9.65

Chemical Name	IP (eV)
(D continued)	
Decaborane	9.88
Diazomethane	9.00
Diborane	12.00
Dibromochloromethane	10.59
Dibromodifluoromethane	11.07
Dibromomethane	10.49
Dibutylamine	7.69
Dichlorodifluoromethane (Freon 12)	12.31
Dichlorofluoromethane	12.39
Dichloromethane	11.35
Diethoxymethane	9.70
Diethyl amine	8.01
Diethyl ether	9.53
Diethyl ketone	9.32
Diethyl sulfide	8.43
Diethyl sulfite	9.68
Difluorodibromomethane	11.07
Dihydropyran	8.34
Diiodomethane	9.34
Diisopropylamine	7.73
Dimethoxymethane (methylal)	10.00
Dimethyl amine	8.24
Dimethyl ether	10.00
Dimethyl sulfide	8.69
Dimethylaniline	7.13
Dimethylformamide	9.18
Dimethylphthalate	9.64
Dinitrobenzene	10.71
Dioxane	9.19
Diphenyl	7.95
Dipropyl amine	7.84
Dipropyl sulfide	8.30
Durene	8.03
m-Dichlorobenzene	9.12
N,N-Diethyl acetamide	8.60
N,N-Diethyl formamide	8.89
N,N-Dimethyl acetamide	8.81
N,N-Dimethyl formamide	9.12
o-Dichlorobenzene	9.06
p-Dichlorobenzene	8.95
p-Dioxane	9.13
trans-Dichloroethene	9.66
E	
Epichlorohydrin	10.20
Ethane	11.65
Ethanethiol (ethyl mercaptan)	9.29
Ethanolamine	8.96
Ethene	10.52
Ethyl acetate	10.11
Ethyl alcohol	10.48
Ethyl amine	8.86
Ethyl benzene	8.76
Ethyl bromide	10.29
Ethyl chloride (chloroethane)	10.98
Ethyl disulfide	8.27
Ethyl ether	9.51
Ethyl formate	10.61
Ethyl iodide	9.33
Ethyl isothiocyanate	9.14
Ethyl mercaptan	9.29
Ethyl methyl sulfide	8.55
Ethyl nitrate	11.22
Ethyl propionate	10.00
Ethyl thiocyanate	9.89
Ethylene chlorohydrin	10.52
Ethylene diamine	8.60
Ethylene dibromide	10.37
Ethylene dichloride	11.05
Ethylene oxide	10.57
Ethylenimine	9.20
Ethynylbenzene	8.82

Chemical Name	IP (eV)
F	
2-Furaldehyde	9.21
Fluorine	15.70
Fluorobenzene	9.20
Formaldehyde	10.87
Formamide	10.25
Formic acid	11.05
Freon 11 (trichlorofluoromethane)	11.77
Freon 112 (1,1,2,2-tetrachloro-1,2-difluoroethane)	11.30
Freon 113 (1,1,2-trichloro-1,2,2-trifluoroethane)	11.78
Freon 114 (1,2-dichloro-1,1,2,2-tetrafluoroethane)	12.20
Freon 12 (dichlorodifluoromethane)	12.31
Freon 13 (chlorotrifluoromethane)	12.91
Freon 22 (chlorofluoromethane)	12.45
Furan	8.89
Furfural	9.21
m-Fluorotoluene	8.92
o-Fluorophenol	8.66
o-Fluorotoluene	8.92
p-Fluorotoluene	8.79
H	
1-Hexene	9.46
2-Heptanone	9.33
2-Hexanone	9.35
Heptane	10.08
Hexachloroethane	11.10
Hexane	10.18
Hydrazine	8.10
Hydrogen	15.43
Hydrogen bromide	11.62
Hydrogen chloride	12.74
Hydrogen cyanide	13.91
Hydrogen fluoride	15.77
Hydrogen iodide	10.38
Hydrogen selenide	9.88
Hydrogen sulfide	10.46
Hydrogen telluride	9.14
Hydroquinone	7.95
I	
1-Iodo-2-methylpropane	9.18
1-Iodobutane	9.21
1-Iodopentane	9.19
1-Iodopropane	9.26
2-Iodobutane	9.09
2-Iodopropane	9.17
Iodine	9.28
Iodobenzene	8.73
Isobutane	10.57
Isobutyl acetate	9.97
Isobutyl alcohol	10.12
Isobutyl amine	8.70
Isobutyl formate	10.46
Isobutyraldehyde	9.74
Isobutyric acid	10.02
Isopentane	10.32
Isophorone	9.07
Isoprene	8.85
Isopropyl acetate	9.99
Isopropyl alcohol	10.16
Isopropyl amine	8.72
Isopropyl benzene	8.69
Isopropyl ether	9.20
Isovaleraldehyde	9.71
m-Iodotoluene	8.61
o-Iodotoluene	8.62
p-Iodotoluene	8.50
K	
Ketene	9.61
L	
2,3-Lutidine	8.85
2,4-Lutidine	8.85
2,6-Lutidine	8.85

Chemical Name	IP (eV)
M	
2-Methyl furan	8.39
2-Methyl naphthalene	7.96
1-Methyl naphthalene	7.96
2-Methyl propene	9.23
2-Methyl-1-butene	9.12
2-Methylpentane	10.12
3-Methyl-1-butene	9.51
3-Methyl-2-butene	8.67
3-Methylpentane	10.08
4-Methylcyclohexene	8.91
Maleic anhydride	10.80
Mesityl oxide	9.08
Mesitylene	8.40
Methane	12.98
Methanethiol (methyl mercaptan)	9.44
Methyl acetate	10.27
Methyl acetylene	10.37
Methyl acrylate	9.90
Methyl alcohol	10.85
Methyl amine	8.97
Methyl bromide	10.54
Methyl butyl ketone	9.34
Methyl butyrate	10.07
Methyl cellosolve	9.60
Methyl chloride	11.28
Methyl chloroform (1,1,1-trichloroethane)	11.00
Methyl disulfide	8.46
Methyl ethyl ketone	9.53
Methyl formate	10.82
Methyl iodide	9.54
Methyl isobutyl ketone	9.30
Methyl isobutyrate	9.98
Methyl isocyanate	10.67
Methyl isopropyl ketone	9.32
Methyl isothiocyanate	9.25
Methyl mercaptan	9.44
Methyl methacrylate	9.70
Methyl propionate	10.15
Methyl propyl ketone	9.39
□-Methyl styrene	8.35
Methyl thiocyanate	10.07
Methylal (dimethoxymethane)	10.00
Methylcyclohexane	9.85
Methylene chloride	11.32
Methyl-n-amyl ketone	9.30
Monomethyl aniline	7.32
Monomethyl hydrazine	7.67
Morpholine	8.20
n-Methyl acetamide	8.90
N	
1-Nitropropane	10.88
2-Nitropropane	10.71
Naphthalene	8.12
Nickel carbonyl	8.27
Nitric oxide, (NO)	9.25
Nitrobenzene	9.92
Nitroethane	10.88
Nitrogen	15.58
Nitrogen dioxide	9.78
Nitrogen trifluoride	12.97
Nitromethane	11.08
Nitrotoluene	9.45
p-Nitrochloro benzene	9.96
O	
Octane	9.82
Oxygen	12.08
Ozone	12.08
P	
1-Pentene	9.50
1-Propanethiol	9.20
2,4-Pentanedione	8.87

Chemical Name	IP (eV)
(P continued)	
2-Pentanone	9.38
2-Picoline	9.02
3-Picoline	9.02
4-Picoline	9.04
n-Propyl nitrate	11.07
Pentaborane	10.40
Pentane	10.35
Perchloroethylene	9.32
Pheneloic	8.18
Phenol	8.50
Phenyl ether (diphenyl oxide)	8.82
Phenyl hydrazine	7.64
Phenyl isocyanate	8.77
Phenyl isothiocyanate	8.52
Phenylene diamine	6.89
Phosgene	11.77
Phosphine	9.87
Phosphorus trichloride	9.91
Phthalic anhydride	10.00
Propane	11.07
Propargyl alcohol	10.51
Propiolactone	9.70
Propionaldehyde	9.98
Propionic acid	10.24
Propionitrile	11.84
Propyl acetate	10.04
Propyl alcohol	10.20
Propyl amine	8.78
Propyl benzene	8.72
Propyl ether	9.27
Propyl formate	10.54
Propylene	9.73
Propylene dichloride	10.87
Propylene imine	9.00
Propylene oxide	10.22
Propyne	10.36
Pyridine	9.32
Pyrrrole	8.20
Q	
Quinone	10.04
S	
Stibine	9.51
Styrene	8.47
Sulfur dioxide	12.30
Sulfur hexafluoride	15.33
Sulfur monochloride	9.66
Sulfuryl fluoride	13.00
T	
o-Terphenyls	7.78
1,1,2,2-Tetrachloro-1,2-difluoroethane (Freon 112)	11.30
1,1,1-Trichloroethane	11.00
1,1,2-Trichloro-1,2,2-trifluoroethane (Freon 113)	11.78
2,2,4-Trimethyl pentane	9.86
o-Toluidine	7.44
Tetrachloroethane	11.62
Tetrachloroethene	9.32
Tetrachloromethane	11.47
Tetrahydrofuran	9.54
Tetrahydropyran	9.25
Thioacetic acid	10.00
Thiophene	8.86
Toluene	8.82
Tribromoethene	9.27
Tribromofluoromethane	10.67
Tribromomethane	10.51
Trichloroethene	9.45
Trichloroethylene	9.47
Trichlorofluoromethane (Freon 11)	11.77
Trichloromethane	11.42
Triethylamine	7.50
Trifluoromonobromo-methane	11.40

Chemical Name	IP (eV)
(T continued)	
Trimethyl amine	7.82
Tripropyl amine	7.23
V	
o-Vinyl toluene	8.20
Valeraldehyde	9.82
Valeric acid	10.12
Vinyl acetate	9.19
Vinyl bromide	9.80
Vinyl chloride	10.00
Vinyl methyl ether	8.93

Chemical Name	IP (eV)
W	
Water	12.59
X	
2,4-Xylidine	7.65
m-Xylene	8.56
o-Xylene	8.56
p-Xylene	8.45

Notes:



PO Box 979
680 Fairfield Court
Ann Arbor, Michigan 48106-0979
734.761.1270 Fax 734.761.3220

Returning an Instrument for Repair

ENMET instruments may be returned to the factory or any one of our Field Service Centers for regular repair service or calibration. The **ENMET** Repair Department and Field Service Centers also perform warranty service work.

When returning an instrument to the factory or service center for service, paperwork must be included which contains the following information:

- A purchase order number or reference number.
- A contact name with return address, telephone and fax numbers
- Specific instructions regarding desired service or description of the problems being encountered.
- Date of original purchase and copy of packing slip or invoice for warranty consideration.
- If a price estimate is required, please note it accordingly *and be sure to include a fax number.*

Providing the above information assists in the expedient repair and return of your unit.

Failure to provide this information can result in processing delays.

ENMET charges a one hour minimum billing for all approved repairs with additional time billed to the closest tenth of an hour. All instruments sent to **ENMET** are subject to a minimum evaluation fee, even if returned unrepared. Unclaimed instruments that **ENMET** has received without appropriate paperwork or attempts to advise repair costs that have been unanswered, after a period of 60 days, may be disposed of or returned unrepared COD with the evaluation fee.

Service centers may have different rates or terms. Be sure to contact them for this information.

Repaired instruments are returned by UPS/FedEx Ground and are not insured unless otherwise specified. If expedited shipping methods or insurance is required, it must be stated in your paperwork.

Note: Warranty of customer installed components.

If a component is purchased and installed in the field, and fails within the warranty term, it can be returned to **ENMET** and will be replaced, free of charge, per **ENMET**'s returned goods procedure.

If the entire instrument is returned to **ENMET** Corporation with the defective item installed, the item will be replaced at no cost, but the instrument will be subject to labor charges at half of the standard rate.



Repair Return Form

Mailing Address:
ENMET Corporation
PO Box 979
Ann Arbor, Michigan 48106

Shipping Address:
ENMET Corporation
Attn: Repair Department
680 Fairfield Court
Ann Arbor, Michigan 48108

Phone Number: 734.761.1270
FAX Number: 734.761.3220

Your Mailing Address:

Your Shipping Address:

Contact Name: _____ **Your Phone:** _____

Your PO/Reference Number: _____ **Your FAX:** _____

Payment Terms: COD

(Check one) **VISA / MasterCard** _____
Card number Expiration Card Code

American Express _____
Card number Expiration Card Code

Name as it appears on the credit card _____

Return Shipping Method:

UPS: Ground 3 Day Select Next Day Air ND Air Saver 2-Day Air
 UPS Account number: _____

Federal Express: Ground Express Saver P-1 Standard 2-Day Air
 FedEx Account number: _____

Would you like ENMET to insure the return shipment?

No Yes **Insurance Amount:** \$ _____